Installationsanleitung

D

Schallwächter, Best.-Nr. 70 0790 00

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GB

Sound detector, order no. 70 0790 00

Sound detector for acoustic monitoring of rooms. Automatic call activation in case of any noises.

Sensitivity and time delay can be adjusted. So it can be adjusted which volume and noise duration shall activate a call.

Note: Normal conversation will not activate a call.

Note! The complete installation of the system is described in the technical manual.

A Connection possibilities

The sound detector can be connected to different connection sockets and patient bedside combinations from the company Tunstall GmbH.

B User instructions

To switch on

The device will be automatically switched on by plugging into the corresponding socket for call devices. It is immediately operational.

Adjustments:

1 Sensitivity (sensitivity of speaking):

This regulator is used for adjusting the sensitivity of sound loudness level. A call will be activated above this level. Scale of threshold value 0 to 9.

Adjustment 9 = highest sensitivity: a slight noise activates a call.

2 Delay:

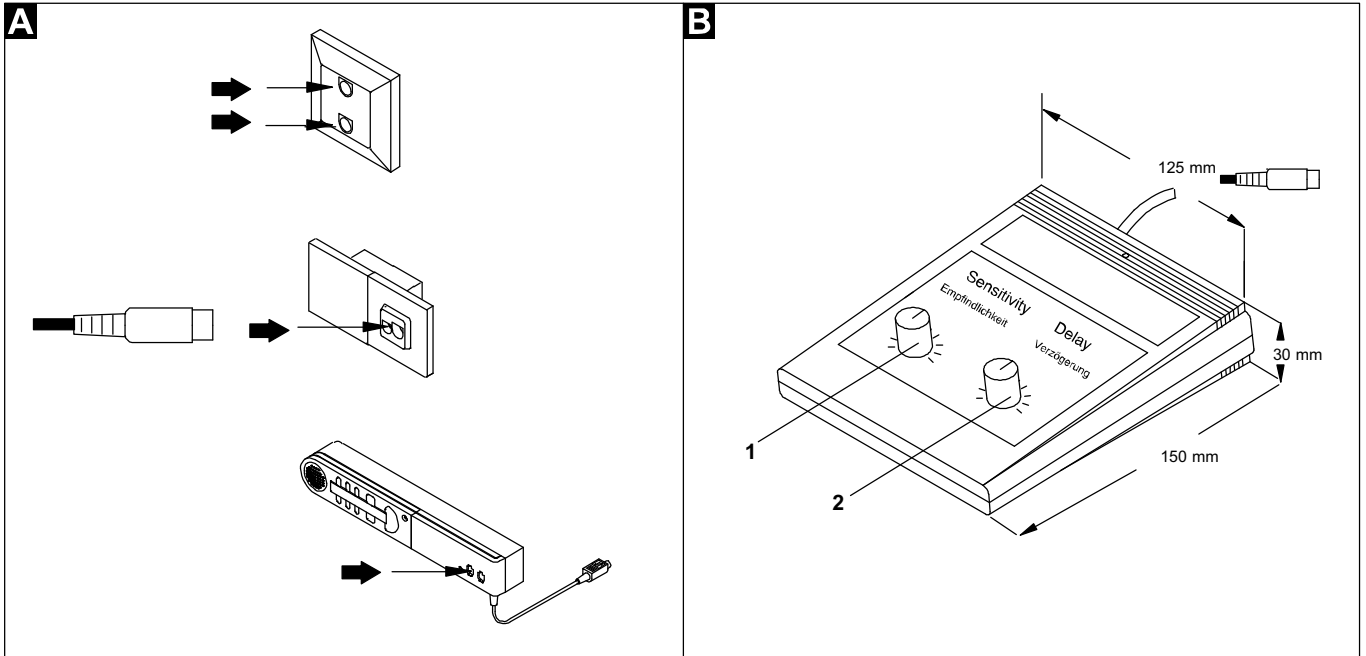
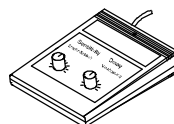
This regulator is used for adjusting the minimum duration of sound, beyond which leads to call activation.

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© Tunstall GmbH, Orkotten 66, D-48291 Telgte, www.tunstall.de