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Connection socket with call switch, order no. 70 0171 03

Connection socket with two 8pole jacks for connection of call devices like pear push switches or medical monitoring devices. The connection for a third call device (external call switch) is made on the back side with a special connection cable.

Note! The complete installation of the system is described in the technical manual.

Attention! The printed circuit board includes electrostatic sensitive components. Avoid touching.

A Mounting

- | | |
|---|--|
| 1* Back box | 4 Cover |
| 2* Connector (70 0807 01) | 5 Fixing clamp (if screw fixing is not possible) |
| 3 Mounting frame with printed circuit board (pre-mounted) | |
- * not included in delivery.

1. Plug the connector 2 into the green socket on the printed circuit board 3.
2. If required: Plug the terminal of the connection cable (50 0308 02) for the external call device into the black socket on the printed circuit board 3.
3. Fix the mounting frame 3 using the screwpoints of the back box 1. If screw fixing is not possible, use the fixing clamp 5 to fix the mounting frame to the box. The fixing clamp cannot be used with back boxes for partition walls.
4. Fit the cover 4 over the mounting frame 3 and lock it.

B Connections

As external call devices only normally-closed contacts may be used. The external call device is connected via the special connection cable, order no. 50 0308 02 (length: 50 cm). The maximum cable length for the external call device is 2.5 m.




C Setting the bed number

The corresponding bed number can be specified by means of the coding switches 1, 2 and 3 of the printed circuit board.

*1 Attention: Diagnostic call only in combination with programmed bed number. With bed number 6 no diagnostic call can be used.

D Dismantling

1. Place the screwdriver in one of the two notches and take off the cover.
2. Remove the screws at the mounting frame 3.
3. Remove the connector 2.

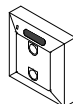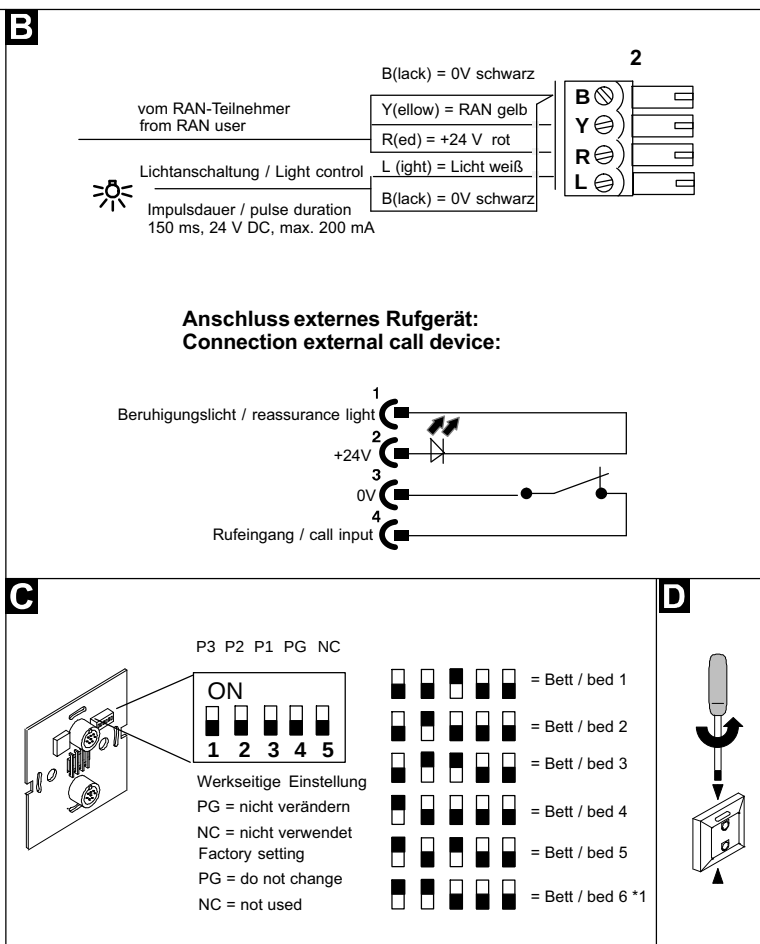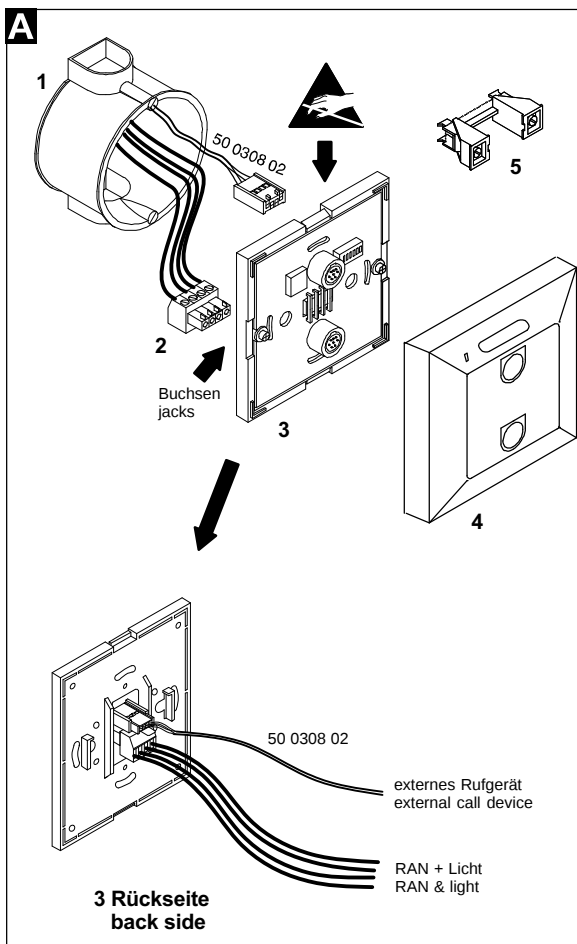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