Steckvorrichtung Kombi Kanal Universal, Best.-Nr. 70 0448 00

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*) When using bed number 6 no diagnostic call can be used.

2. Light option

Jumper open: Separate outputs LT1 and LT2 for the light control / light relays (light switches are separate).

Jumper placed: Outputs LT1 and LT2 are connected (light switches are parallel). In this case connect only one light control / light relay.

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1. Open the housing of the medical supply unit.
2. Put in the connection socket.
3. Put on the flat band cable to the medical supply unit.
4. If there is readinglight available put on also the connection to the light control/light relays.
5. Close the housing of the medical supply unit.

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





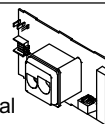
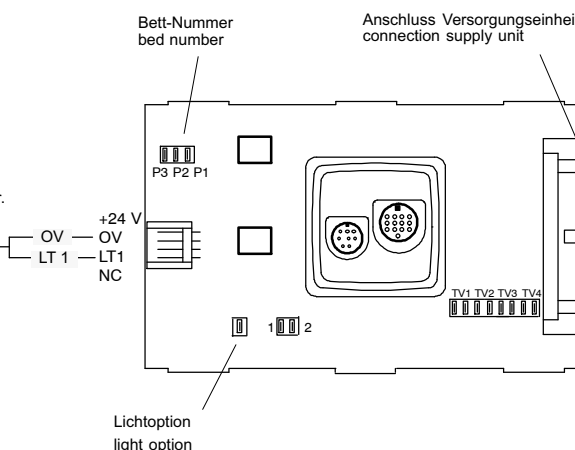
24 V signal for the length of time the light switch remains pressed. The output may be loaded with 60 mA. Non-dimmable and dimmable light same as for room light, refer to next page.

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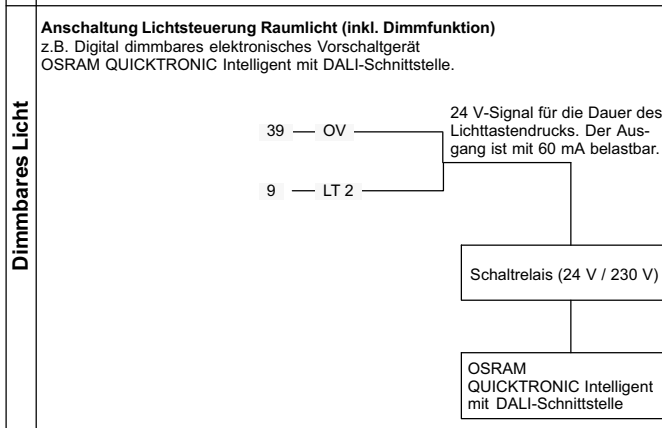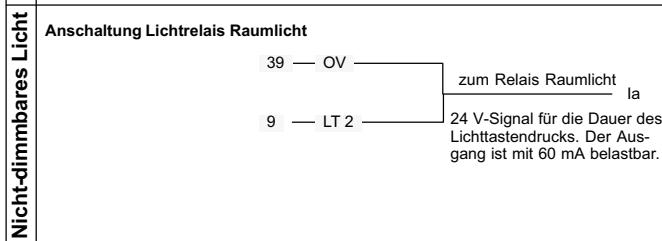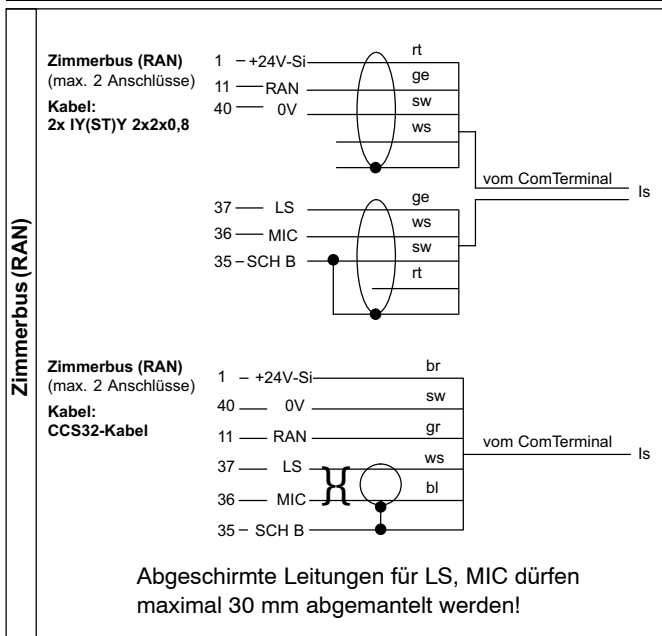Connections to connection field GB

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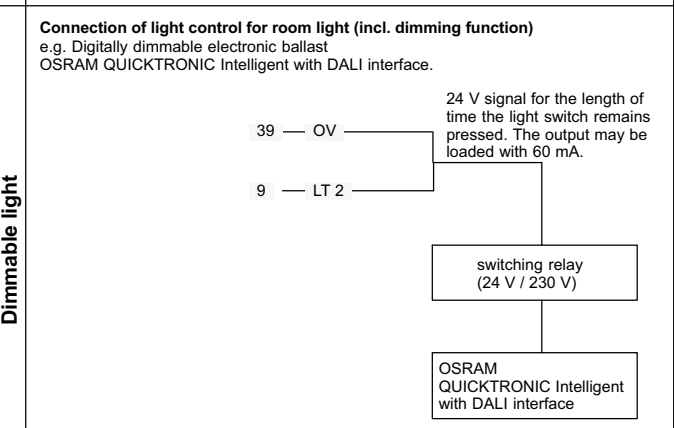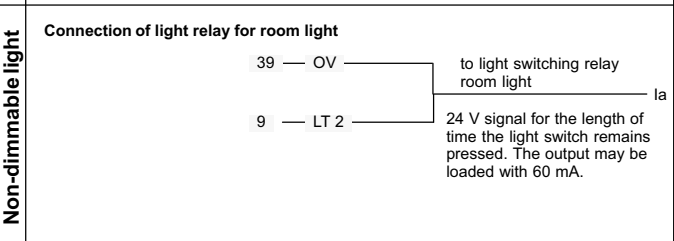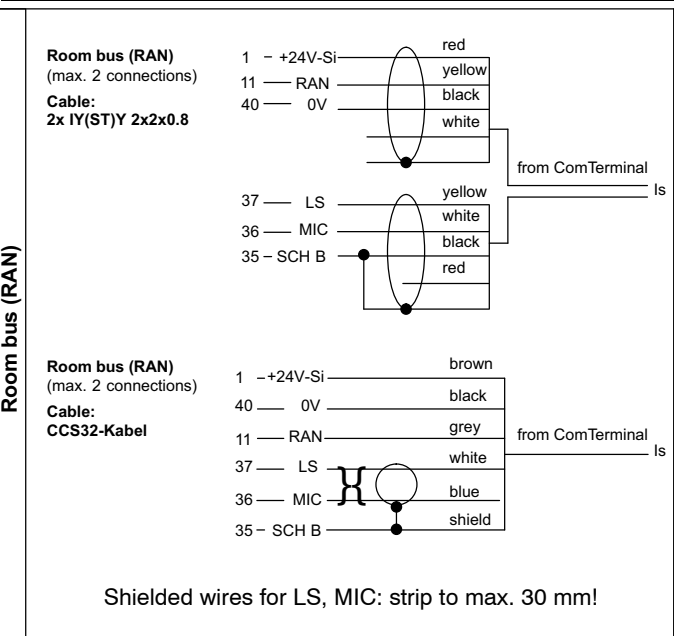Connection socket combi bedhead unit universal, order no. 70 0448 00

Combined connection socket with two different sockets for connection of patient handsets and other call devices. Designed for transmission of calls and for routing of control signals for external actuators.

Use only in combination with patient handset universal, order no. 74 0748 00, and RAN interface universal, order no. 70 0848 00.

Suitable for mounting in medical supply units.

Note! The complete installation of the system is described in the technical manual.

Attention! The printed circuit board includes electrostatic sensitive components. Avoid touching.

Electromagnetic compatibility! Due to insufficient suppression at fluorescent lamps in medical supply units faults in the call system may occur. These external interferences possibly can be avoided by installing suppressor elements (varistor circuits). Varistor circuits are commercially available from electric/electronic suppliers. Tunstall offers a proven over-voltage filter 230 V (order no. 70 0890 97). Medical supply units may differ clearly in EMC behaviour. Even two supply units of the same type may differ, if the installation is different.

When laying call system cables in medical supply units the regulations of European standard ISO 11197 have to be followed.

A Setting the connection socket

Before mounting the connection socket please adjust as follows:

1. Setting of bed number

Bed no.	jumper placed:
Bed 1	P1
Bed 2	P2
Bed 3	P1, P2 (factory setting)
Bed 4	P3
Bed 5	P1, P3
Bed 6*)	P2, P3

*) When using bed number 6 no diagnostic call can be used.

2. Light option

Jumper open: Separate outputs LT1 and LT2 for the light control / light relays (light switches are separate).
Jumper placed: Outputs LT1 and LT2 are connected (light switches are parallel). In this case connect only one light control / light relay.

■ Mounting

After the settings are made the connection socket is mounted into the medical supply unit. The mounting depends on the individual model of the medical supply unit.

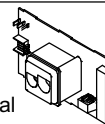
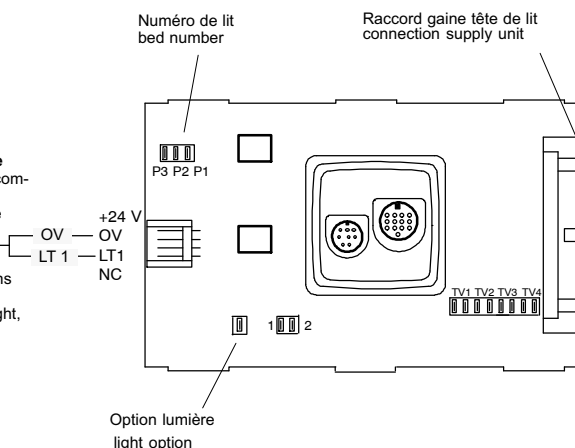
- Open the housing of the medical supply unit.
- Put in the connection socket.
- Put on the flat band cable to the medical supply unit.
- If there is readinglight available put on also the connection to the light control/light relays.
- Close the housing of the medical supply unit.

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Connection light control for reading light

24 V signal for the length of time the light switch remains pressed. The output may be loaded with 60 mA.
Non-dimmable and dimmable light same as for room light, refer to next page.

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
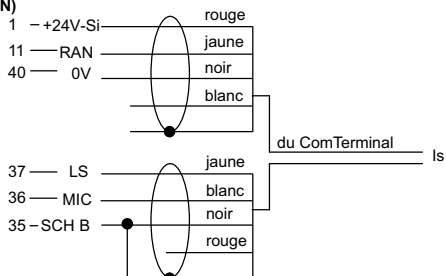
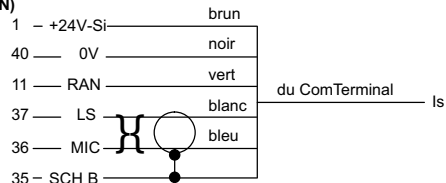
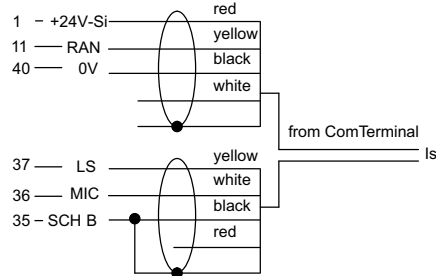
Room bus (RAN)
(max. 2 connections)
Cable:
2x IY(ST)Y 2x2x0.8

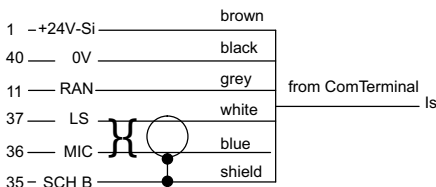
Room bus (RAN)
(max. 2 connections)
Cable:
CCS32-Kabel

Shielded wires for LS, MIC: strip to max. 30 mm!

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
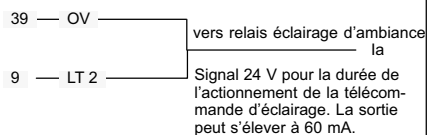
Non-dimmable light

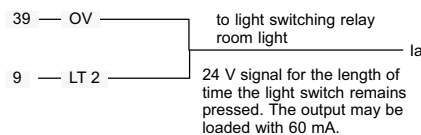Connection of light relay for room light

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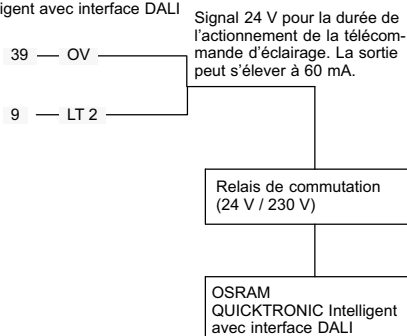
Dimmable light

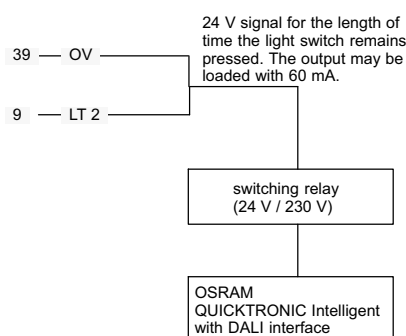Connection of light control for room light (incl. dimming function)
e.g. Digitally dimmable electronic ballast
OSRAM QUICKTRONIC Intelligent with DALI interface.

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