Steckvorrichtung Kombi Universal, Best.-Nr. 70 0449 00

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Connection socket combi universal, order no. 70 0449 00

Combined connection socket with two different sockets for connection of patient handsets and other call devices. Designed for transmission of calls and for routing of control signals for external actuators. Suitable for wall mounting.

Use only in combination with patient handset universal, order no. 74 0748 00, and RAN interface universal, order no. 70 0848 00.

Note! The complete installation of the system is described in the technical manual.

Attention! The printed circuit board includes electrostatic sensitive components. Avoid touching.

A Mounting

1. Fit the socket **1** into the back box.
2. Screw the mounting frame **2** together with the socket onto the back box.
3. Connect both plugs **3** with the sockets (A in A, B in B).
4. Push the cover **4** onto the mounting frame **2**.

B Connection

- Useable wire gauges: 0.55 - 1 mm Ø
- Shielded wires for LS, MIC: Strip to max. 30 mm!
- Insulate the guide wire (SCH) to prevent short circuiting.
- Connect only one wire per terminal.

Handling of terminals with solderless connection technique:

1. Strip the wire (8 mm).
2. Insert the stripped wire into the terminal until it stops.
3. For detaching the wire insert a screw driver with a tip of 2.5 x 0.4 mm or 3.5 x 0.5 mm into the opening and pull out the wire.

Note! For connecting the shield wire, use the screw driver to hold the spring contact open, while you are inserting the shield wire. Afterwards check for a firm wire contact by pulling at the wire.


C Setting bed number

Setting of bed numbers 1 to 6: Use switches P3, P2, P1 as shown in the table.

Example: Bed number 1 = P3 OFF, P2 OFF, P1 ON


PG and TV switches: Do not change.

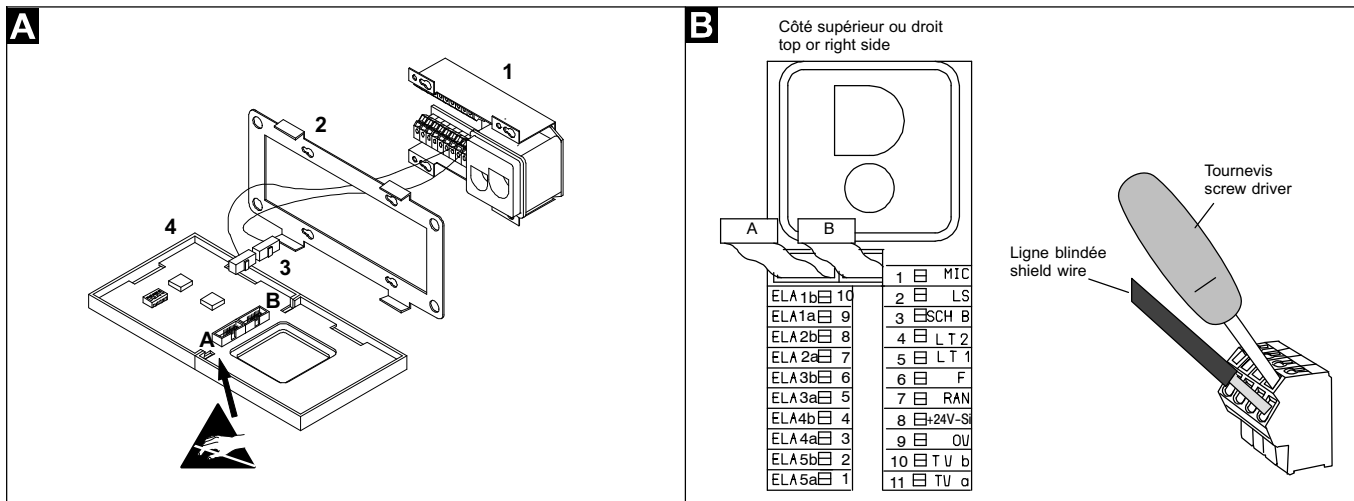*1 Attention: With bed number 6 diagnostic call is not possible.

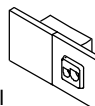	P3	P2	P1
1	off	off	on
2	off	on	off
3	off	on	on
4	on	off	off
5	on	off	on
6	on	on	off

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