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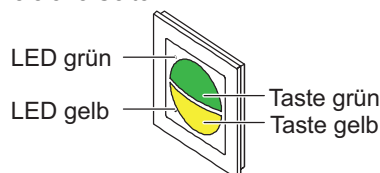
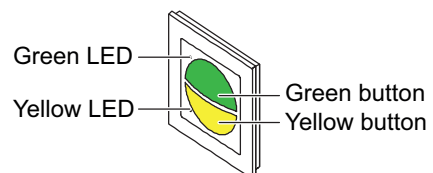
**Staff presence switch, order no. 77 0212 00...\*)**

Two presence buttons for switching on and off the staff presence for two different staff groups. The LEDs are on, when the respective presence is switched on (reminder light).

- Presence button (green) with LED (green) for staff 1.

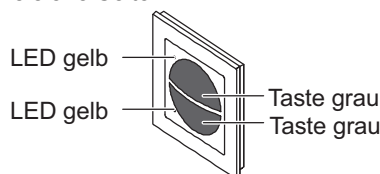
- Presence button (yellow) with LED (yellow) for staff 2.

“Call tone” function available, but disabled in the factory setting; for details refer to page 4.

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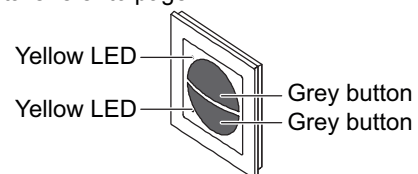
**Cancel switch/WC, order no. 77 0213 00...\*)**

Two cancel buttons (grey) for cancelling WC calls and WC emergency calls. The LEDs (yellow) are on, when a WC call or a WC emergency is raised that can be cancelled with these buttons (reminder light).

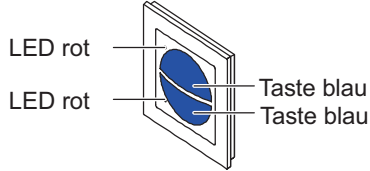
“Call tone” function available, but disabled in the factory setting; for details refer to page 4.

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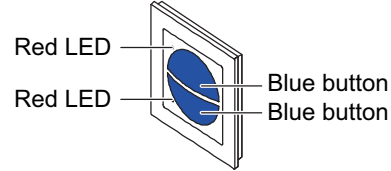
Two alarm buttons (blue) for raising cardiac alarms, if staff presence is switched on in the room. When the alarm switch is active, the LEDs (red) are on with a weak light for finding the buttons in the dark (night location light). The LEDs are on with a bright light, when an alarm was raised (reassurance light).

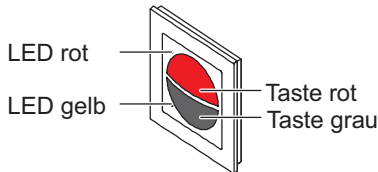
Optional setting: Cardiac alarms can also be raised, if no staff presence is switched on in the room.

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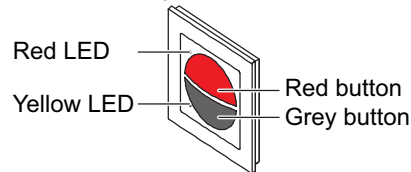
**Call switch/WC with cancel switch, 77 0217 00...\*)**

Call button (red) for raising WC calls. The LED (red) is on with a weak light for finding the button in the dark (night location light). The LED is on with a bright light, when a WC call was raised (reassurance light).

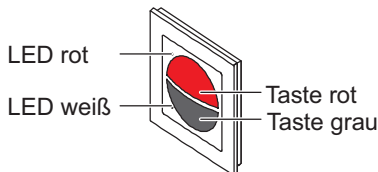
Cancel button (grey) for cancelling WC calls and WC emergency calls. The LED (yellow) is on, when a WC call or a WC emergency is raised that can be cancelled with this button (reminder light).

“Call tone” function available, but disabled in the factory setting; for details refer to page 4.

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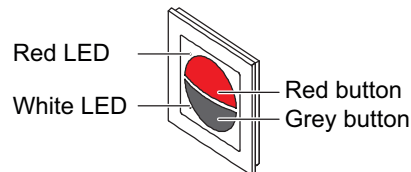
**Call switch with privacy switch, 77 0218 00...\*)**

Call button (red) for raising calls. The LED (red) is on with a weak light for finding the button in the dark (night location light). The LED is on with a bright light, when a call was raised (reassurance light).

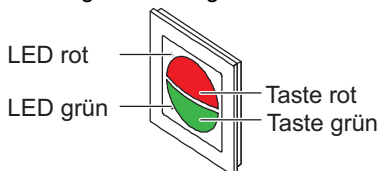
Privacy button (grey) for switching on and off the privacy in the room, if staff presence is switched on in the room. The LED (white) is on, if the privacy was switched on with the privacy button.

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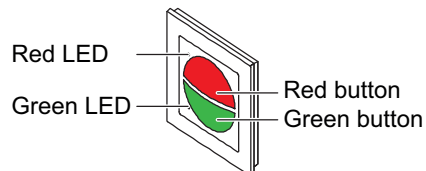
**Staff presence combination with call tone, 77 0219 00...\*)**

Call button (red) for raising calls. The LED (red) is on with a weak light for finding the button in the dark (night location light). The LED is on with a bright light, when a call was raised (reassurance light).

Presence button (green) for switching on and off the staff presence of staff group 1. The LED (green) is on, when presence 1 is switched on (reminder light).

“Call tone” function enabled: The integrated beeper signals forwarded calls, if staff presence is switched on in the room. The beeper also attracts attention for announcements.

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**A Mounting the patress**

Connect the 3-pole connector [4] to the room bus RAN (without speech) according to Fig. A.

**Risk of malfunctions in the nurse call system!**

- The cable shield and the shield wire of the RAN room bus (without speech) are not connected and must be removed in the back box up to the cable sheath.
- The unused wires must not be interconnected.

**B Mounting**

Wall mounting on 1-gang back box (see Fig. B):

1. Screw the support ring [7] to the back box [8] with the back box screws [5]. The openings [6] in the support ring [7] for the locking hooks [2] of the switch module [1] must be located on the top and on the bottom.
2. Lead the connected connector [4] through the support ring [7] out of the back box [8].
3. Place the switch module [1] in the frame [3]. The LEDs of the switch module [1] must be located on the left side.
4. Plug the connected connector [4] to the socket on the rear side of the switch module [1].
5. Press the switch module [1] together with the frame [3] onto the support ring [7] until it engages.

**Dismantling**

1. Lever the switch module [1] together with the frame [3] off the support ring [7].
2. Remove the connector [4] from the switch module [1].
3. Undo the screws [5] of the support ring [7].
4. Remove the support ring [7].

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**NOTE!** The complete installation of the system is described in the Technical Manual.

## Enabling the “Call tone” function

The „call tone” function is available for the following switches:

- Staff presence switch (77 0212 00...)
- Cancel switch/WC (77 0213 00...)
- Call switch/WC with cancel switch (77 0217 00...)
- Staff presence combination with call tone (77 0219 00...)

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This function is only enabled in the factory setting of the staff presence combination with call tone. For all other switches it is disabled.

**CAUTION!** Calls from other rooms must be signalled with a call forwarding tone in rooms with presence activated. If the room terminal is not able to sound call forwarding tones (e.g. ControlTerminal) a switch must sound the call forwarding tones (e.g. staff presence combination with call tone).

### This is how you enable the “call tone” function

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1. Press and hold one button of the switch for approx. 10 seconds until the upper LED lights up yellow. (Sequence: After approx. 5 seconds the switch emits a beep and the upper LED lights up pink. After a further 5 seconds the switch emits a second beep and the upper LED lights up yellow. Now release the button.)

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- Call tone disabled = Upper LED flashes once in red.
- Call tone enabled = Upper LED flashes once in green.

2. By pressing one button repeatedly the setting switches between Green (for “Call tone enabled”) and Red (for “Call tone disabled”). Press the button until the desired setting.

3. To save the setting press and hold one button for approx. 5 seconds until the switch emits a beep.

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## Enabling the optional setting

For the following switches you can enable the optional setting instead of the factory setting, refer to the switch description:

- Call switch (77 0211 00...)
- Call switch/WC (77 0211 01...)
- Cardiac alarm switch (77 0214 00...)

On the next page is described, how you enable the optional setting.

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**This is how you enable the optional setting**

Within the first 10 seconds after the power supply is switched on, you have to start with the following procedure. (Alternatively, the connector of the switch can be pulled off briefly and then re-plugged.)

1. Press and hold one button of the switch for approx. 10 seconds until the upper LED lights up white. (Sequence: After approx. 5 seconds the switch emits a beep and the upper LED lights up pink. After a further 5 seconds the switch emits a second beep and the upper LED lights up white. Now release the button.)

The upper LED signals the current setting:

- Factory setting = Upper LED flashes once in red.
  - Optional setting = Upper LED flashes once in green.
2. By pressing one button repeatedly the setting switches between Green (for optional setting) and Red (for factory setting). Press the button until the desired setting.
  3. To save the setting press and hold one button for approx. 5 seconds until the switch emits a beep.

The new setting is saved. The upper LED signals the saved setting.

NOTE! If you do not press a button within 30 seconds, the new setting is not saved.

**Setting a channel number (0 – 7)**

If functional units are requested, the switches have to be assigned to different channels. Refer to your installation plan to find out, whether channel numbers have to be assigned. Factory set to channel 0.

Within the first 10 seconds after the power supply is switched on, you have to start with the following procedure. (Alternatively, the connector of the switch can be pulled off briefly and then re-plugged.)

1. Press and hold one button of the switch for approx. 5 seconds until the switch emits a beep and the upper LED lights up pink.

The upper LED signals the current setting:

- Channel 0 = Upper LED flashes once in red (factory-set).
  - Channel 1 = Upper LED flashes once in green.
  - Channel 2 = Upper LED flashes twice in green.
  - and so on ...
  - Channel 7 = Upper LED flashes seven times in green.
2. Set the channel number predefined in your installation plan as follows:
    - Channel 0 = Do not press a button.
    - Channel 1 = Press one button once.
    - Channel 2 = Press one button twice.
    - and so on ...
    - Channel 7 = Press one button seven times.

3. To save the setting press and hold one button of the switch until the switch emits a beep.

The new setting is saved. The upper LED signals the saved setting.

NOTE! If you do not press a button within 30 seconds, the new setting is not saved.

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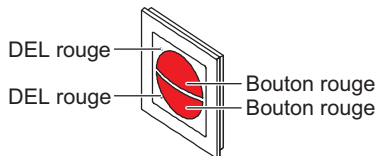
## Presentation of the switches

...\*) The letter at the end of the order number represents the frame of the switch:

**A** = Frame dimensions (HxW): 91 x 91 mm.

**F** = Frame dimensions (HxW): 80 x 80 mm.

**C** = Frame dimensions (HxW): 107 x 107 mm.

CAUTION! The frame C is made from real glass!

## Call switch, order no. 77 0211 00...\*)

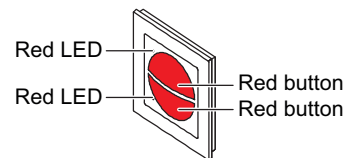
Two call buttons (red) for raising calls. The LEDs (red) are on with a weak light for finding the buttons in the dark (night location light). The LEDs are on with a bright light, when a call was raised (reassurance light).

Optional setting: WC calls are raised instead of calls.

## Call switch/WC, order no. 77 0211 01...\*)

same as 77 0211 00..., but for WC call.

Optional setting: Calls are raised instead of WC calls.

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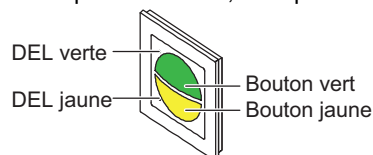
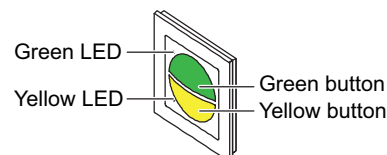
- Presence button (green) with LED (green) for staff 1.
- Presence button (yellow) with LED (yellow) for staff 2.

“Call tone” function available, but disabled in the factory setting; for details refer to page 9.

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### Enabling the “Call tone” function

The “call tone” function is available for the following switches:

- Staff presence switch (77 0212 00...)
- Cancel switch/WC (77 0213 00...)
- Call switch/WC with cancel switch (77 0217 00...)
- Staff presence combination with call tone (77 0219 00...)

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Only in the factory setting of the staff presence combination with call tone the “call tone” is enabled. For all other switches it is disabled.

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#### This is how you enable the “call tone” setting

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- Call tone disabled = Upper LED flashes once in red.
  - Call tone enabled = Upper LED flashes once in green.
2. By pressing one button repeatedly the setting switches between Green (for “Call tone enabled”) and Red (for “Call tone disabled”). Press the button until the desired setting.
  3. To save the setting press and hold one button for approx. 5 seconds until the switch emits a beep.

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**NOTE!** If you do not press a button within 30 seconds, the new setting is not saved.

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- Channel 0 = Upper LED flashes once in red (factory-set).
- Channel 1 = Upper LED flashes once in green.
- Channel 2 = Upper LED flashes twice in green.
- and so on ...
- Channel 7 = Upper LED flashes seven times in green.

2. Set the channel number predefined in your installation plan as follows:

- Channel 0 = Do not press a button.
- Channel 1 = Press one button once.
- Channel 2 = Press one button twice.
- and so on ...
- Channel 7 = Press one button seven times.

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© Tunstall GmbH, Orkotten 66, 48291 Teigte, Germany, www.tunstall.de