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
...\*) The letter at the end of the order number represents the frame of the switch:

**A** = Frame dimensions (HxW): 91 x 91 mm, **F** = Frame dimensions (HxW): 80 x 80 mm

**C** = Frame dimensions (HxW): 107 x 107 mm. **Caution!** The frame **C** is made from real glass!

**Note!** The complete installation of the system is described in the Technical Manual.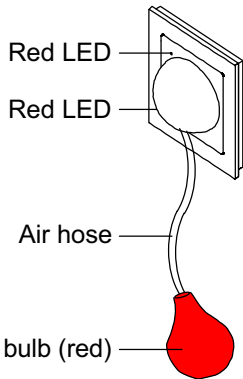

**Pneumatic call switch, order no. 77 0216 00...\*)**


2 m air hose and red rubber bulb for raising calls or WC calls by squeezing the bulb (factory setting: Calls). The LEDs (red) are on with a weak light for finding the pneumatic call switch in the dark (night location light). The LEDs are on with a bright light, when a call was raised (reassurance light).

**Pneumatic call switch/WC, order no. 77 0216 01...\*)**

same as order no. 77 0216 00..., but factory setting: WC call.

**Connecting the connector**

Connect a 3-pole connector (order no. 70 0807 00) to the room bus RAN (without speech) according to Fig. A.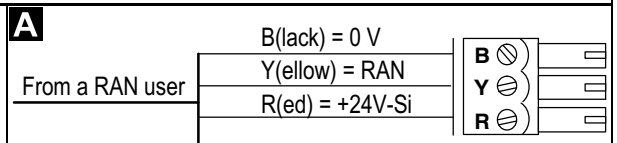

**Mounting**


Wall mounting on 1-gang back box (see Fig. B):

1. Screw the support ring 7 to the back box 8\* with the back box screws 5\*. The openings 6 in the support ring 7 for the locking hooks 2 of the switch module 1 must be located on the top and on the bottom, see Fig. B.
2. Lead the connected connector 4\* through the support ring 7 out of the back box 8\*.
3. Place the switch module 1 in the frame 3. The LEDs of the switch module 1 must be located on the left side, see Fig. B.
4. Plug the connected connector 4\* to the socket on the rear side of the switch module 1.
5. Press the switch module 1 together with the frame 3 onto the support ring 7 until it engages.

**Warning!** Call activation might be prevented. Do not kink the air hose.

**Dismantling**

1. Lever the switch module 1 together with the frame 3 off the support ring 7.
2. Remove the connector 4\* from the switch module 1.
3. Undo the screws 5\* of the support ring 7.
4. Remove the support ring 7.

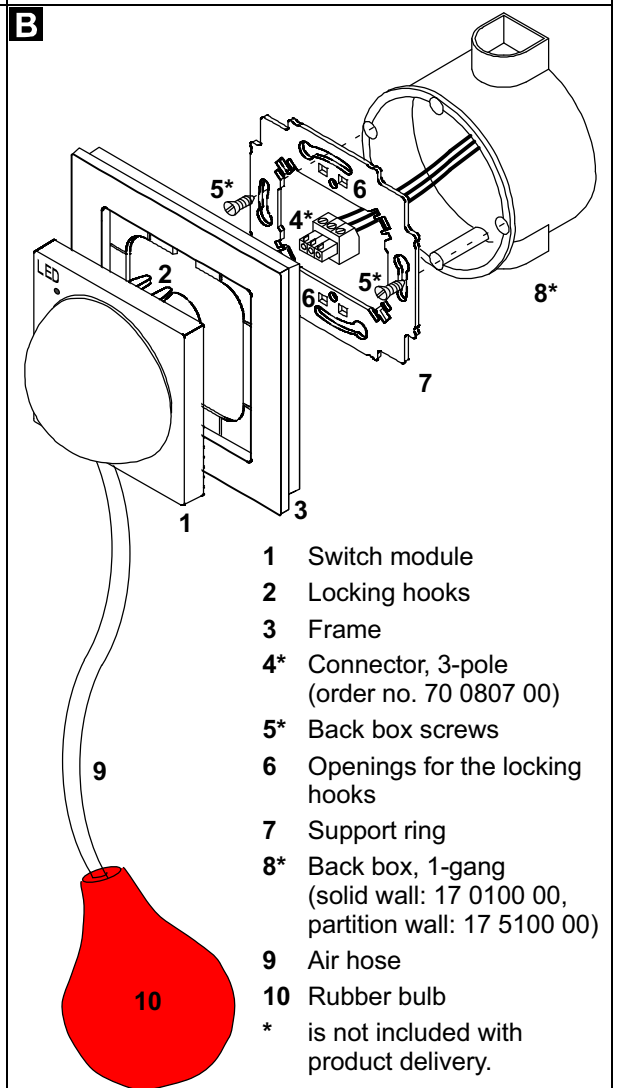- 1 Switch module
- 2 Locking hooks
- 3 Frame
- 4\* Connector, 3-pole (order no. 70 0807 00)
- 5\* Back box screws
- 6 Openings for the locking hooks
- 7 Support ring
- 8\* Back box, 1-gang (solid wall: 17 0100 00, partition wall: 17 5100 00)
- 9 Air hose
- 10 Rubber bulb
- \* is not included with product delivery.

**Setting a channel number (0–7)**

If functional units are requested, the pneumatic call switches have to be assigned to different channels. Refer to your installation plan to find out, whether channel numbers have to be assigned. Factory set to channel 0.

To set a channel number: Within the first ten seconds after the power supply is switched on, you have to start with the procedure described in the following (Alternatively, the connector of the switch can be pulled off briefly and then re-plugged):

1. Squeeze and keep squeezed the rubber bulb for approx. 5 seconds until the switch emits a beep and the upper LED lights up pink.
  - ✓ The upper LED signals the current channel number:
    - Channel 0 = Upper LED flashes once in red.
    - Channel 1 = Upper LED flashes once in green.
    - Channel 2 = Upper LED flashes twice in green.
    - and so on ...
    - Channel 7 = Upper LED flashes seven times in green.
2. Set the channel number predefined in your installation plan as follows:
  - Channel 0 = Do not squeeze the rubber bulb.
  - Channel 1 = Squeeze the rubber bulb once.
  - Channel 2 = Squeeze the rubber bulb twice.
  - and so on ...
  - Channel 7 = Squeeze the rubber bulb seven times.
3. To save the setting squeeze and keep squeezed the rubber bulb until the switch emits a beep.
  - ✓ The new setting is saved.
  - Note! If you do not squeeze the rubber bulb within 30 seconds, the new setting is not saved.
  - ✓ The upper LED signals the set channel number:
    - Channel 0 = Upper LED flashes once in red.
    - Channel 1 = Upper LED flashes once in green.
    - Channel 2 = Upper LED flashes twice in green.
    - and so on ...
    - Channel 7 = Upper LED flashes seven times in green.

**Setting the call type (if needed)**

You can set the call type to be initiated by the switch.

- Pneumatic call switch, order no. 77 0216 00...; factory setting: Call. Optional setting: WC call.
- Pneumatic call switch/WC, order no. 77 0216 01...; factory setting: WC call. Optional setting: Call

To set the call type: Within the first ten seconds after the power supply is switched on, you have to start with the procedure described in the following (Alternatively, the connector of the switch can be pulled off briefly and then re-plugged):

1. Squeeze and keep squeezed the rubber bulb for approx. 10 seconds until the upper LED lights up white. (Sequence: After approx. 5 seconds the switch emits a beep and the upper LED lights up pink. After a further 5 seconds the switch emits a second beep and the upper LED lights up white. Now release the rubber bulb.)
  - ✓ The upper LED signals the current setting for the call type:
    - Factory setting = Upper LED flashes once in red.
    - Optional setting = Upper LED flashes once in green.
2. Set the call type as follows:
  - Factory setting = Do not squeeze the rubber bulb.
  - Optional setting = Squeeze the rubber bulb once.
3. To save the setting squeeze and keep squeezed the rubber bulb until the switch emits a beep.
  - ✓ The new setting is saved.
  - Note! If you do not squeeze the rubber bulb within 30 seconds, the new setting is not saved.
  - ✓ The upper LED signals the set call type:
    - Factory setting = Upper LED flashes once in red.
    - Optional setting = Upper LED flashes once in green.

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