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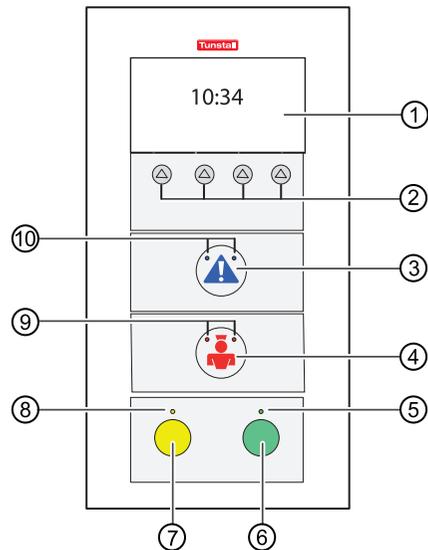
- ① **Display**
- ② **Function keys (situation-related functions)**
- ③ **Blue alarm button**
Blue location light (10) faintly illuminated for finding the button in the dark. Flashing brightly when an alarm was raised at the RoomTerminal (reassurance light).
- ④ **Red call button**
Red location light (9) faintly illuminated for finding the button in the dark. Flashing brightly when a call was raised at the RoomTerminal (reassurance light).
- ⑥ **Green presence button for staff 1**
Green reminder light (5) illuminates when presence 1 is activated.
- ⑦ **Yellow presence button for staff 2**
Yellow reminder light (7) illuminates when presence 2 is activated.

RoomTerminal Flamenco®

Order no. 77 0520 00

User instructions

To log as present

Reporting the current presence of staff at all times is prerequisite for the efficient use of the call system. There are two categories of staff: Staff 1 (e.g. nursing staff), Staff 2 (e.g. doctors).

Activated presence buttons

- indicate the presence of staff in the room,
- acknowledge a fresh call in this room,
- prepare the reception of forwarded calls,
- prepare the initiation of emergency calls. That means, calls from rooms with activated "Presence" buttons are indicated with higher priority,
- put the blue alarm buttons into operation (at the RoomTerminal and at remote alarm switches). That means, pressing a blue alarm button will only initiate a cardiac alarm when staff presence is activated,
- If an announcement is broadcasted at the nurse call system loudspeakers, an attention tone sounds at the RoomTerminal where presence is activated.

When entering the room

- Activate the green presence button (Staff 1) or yellow presence button (Staff 2) at the RoomTerminal or at remote presence switch.
- ✓ The reminder lights of all presence buttons in the room for this staff category come on.

When leaving the room

- De-activate the green presence button (Staff 1) or yellow presence button (Staff 2) at the RoomTerminal or at remote presence switch.
- ✓ The reminder lights of all presence buttons for this staff category in the room go out.

Display of calls

If a call is not acknowledged at the nurse station within a programmed time frame (e.g. 30 seconds), this call will be forwarded to all rooms where the presence is activated.

Forwarded calls are presented in the RoomTerminal's display, and an acoustic signal will sound:

Diagnostic Call
Room: 207

Normal call: Short tone, long pause, short tone, ...
Emergency call: Tone, Pause, Tone, ...
Cardiac alarm: Long tone, short pause, long tone, ...

Providing help to the caller

A call is displayed at the RoomTerminal:

1. Press the function key to acknowledge the call.
✓ The call is acknowledged.
2. Log-out from the current room by de-activating the presence button.
3. Go to the calling location.
4. Log-in at the calling location by activating the presence button.
5. Provide the required help to the calling person.
6. Cancel the call: Automatic cancelling of calls after de-activating the presence button (except calls from WC). For calls from the WC: Press the grey cancel button in the WC or - if the function key available at the RoomTerminal - press this function key.
7. De-activate the presence button.

Note: If an acknowledged call is not cancelled within a programmed time frame (e.g. 3 minutes), it will be re-activated automatically.

Calling for assistance

Raising an emergency call

You are in the patient room and you need assistance:

- Press the red call button at the RoomTerminal or another call switch in the room.
- ✓ The light for the call button is brightly illuminated or is flashing brightly (RoomTerminal). If the presence button is also activated, an emergency call is raised.

Note: An emergency call or cardiac alarm re-activates a previously acknowledged call and it must be answered again before it can be cancelled.

Raising a cardiac alarm

Warning: Alarm buttons are operational only with activated staff presence.

You are in the patient room and you need top priority assistance (e.g. cardiac team):

- Press the blue alarm button at the RoomTerminal or another alarm switch in the room.
- ✓ The light for the alarm button is brightly illuminated or is flashing brightly (RoomTerminal).