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**Connection socket with call switch, bedhead unit, order no. 70 0171 50**

Connection socket with two 8-pole sockets for connection of call devices like pear push switches or medical monitoring devices. The connection of an additional external call device incl. location light and reassurance light is made on the back side with a special connection cable. Two outputs for light control. Mounting in a medical supply unit.

**Note!** The complete installation of the system is described in the technical manual.

**Attention!** The printed circuit board includes electrostatic sensitive components. Avoid touching.

**Electromagnetic compatibility!** Due to insufficient suppression at fluorescent lamps in medical supply units faults in the call system may occur. These external interferences possibly can be avoided by installing suppressor elements (varistor circuits). Varistor circuits are available from the manufacturers of the medical supply units. Tunstall offers a proven overvoltage filter 230 V (order no. 70 0890 97).

Medical supply units may differ clearly in EMC behaviour. Even two supply units of the same type may differ, if the installation is different.

When laying call system cables in medical supply units the regulations of European standard ISO 11197 have to be followed.

**Codierschalter einstellen**

**Bett-Nr.:**

ON	1	2	3	= Bett 1
ON	1	2	3	= Bett 2
ON	1	2	3	= Bett 3
ON	1	2	3	= Bett 4
ON	1	2	3	= Bett 5
ON	1	2	3	= Bett 6 *1

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**Setting coding switches**

**Bed no.:**

ON	1	2	3	= Bed 1
ON	1	2	3	= Bed 2
ON	1	2	3	= Bed 3
ON	1	2	3	= Bed 4
ON	1	2	3	= Bed 5
ON	1	2	3	= Bed 6 *1

**External call device:**

ON	5	6	= No external call device connected.
ON	5	6	= Normally open contact.
ON	5	6	= Normally closed contact.

Coding switch 4 must always be set to OFF.

\*1 Attention: A diagnostic call can only be used, if a bed number is set. With bed number 6 no diagnostic call can be used.

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
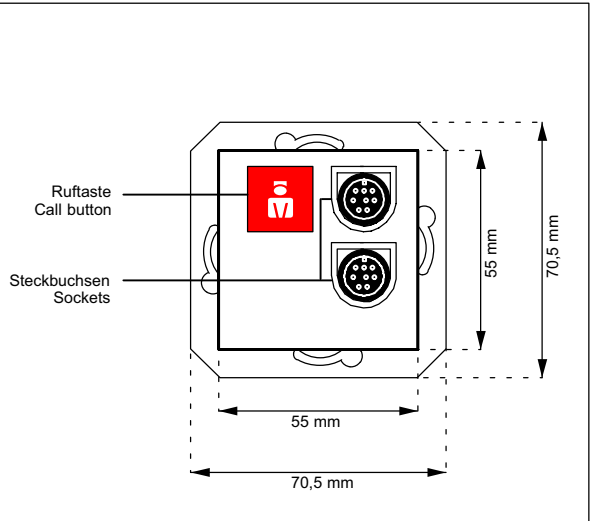
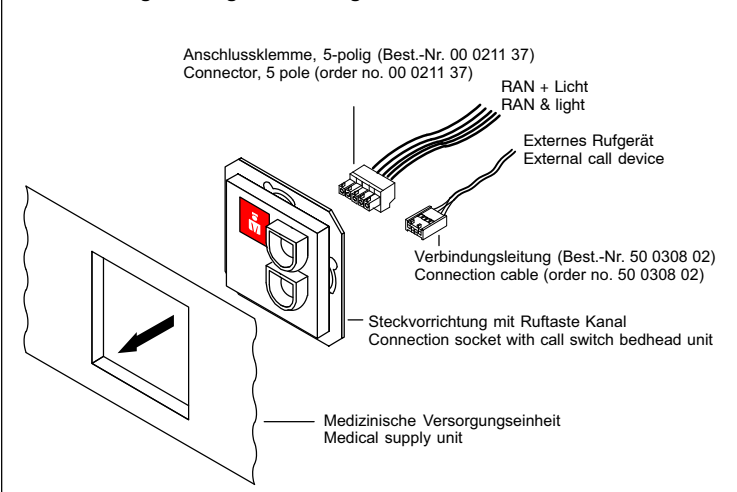
**Mounting**

The mounting depends on the individual model of the medical supply unit. The connection socket is mounted by the manufacturer of the medical supply unit.

**Rückwärtige Montage: / Mounting from backside:**

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
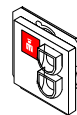
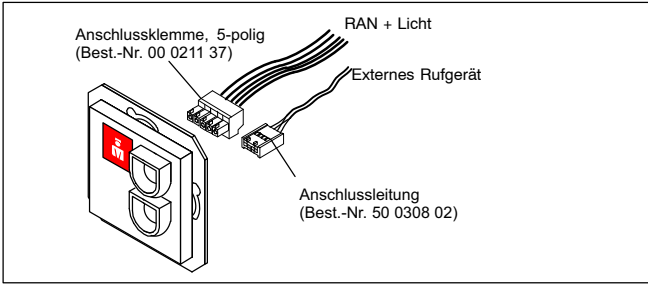
# Connections

GB

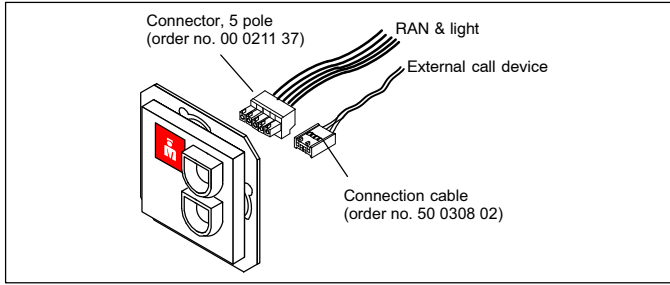
As external call device a normally-closed or a normally-open contact may be used. The external call device is connected via the special connection cable, order no. 50 0308 02 (length: 50 cm). The maximum cable length for the external call device is 2.5 m.

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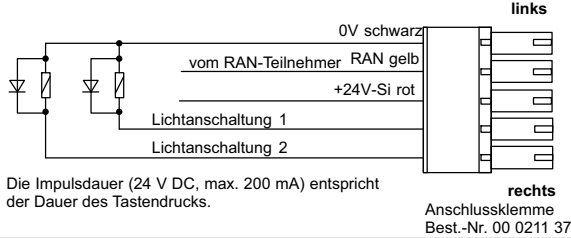
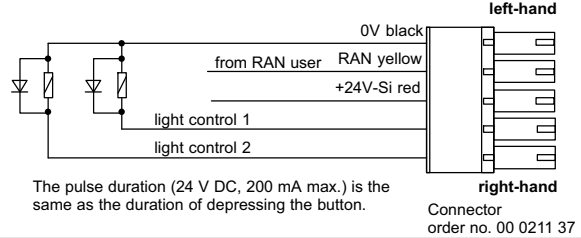
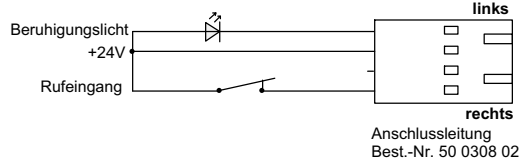
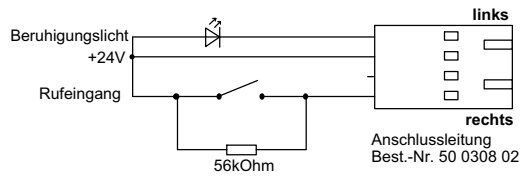
## Connecting external call device as normally closed contact:
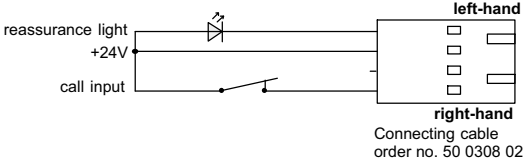
## Connecting external call device as normally open contact (e.g. Siemens Mediset):

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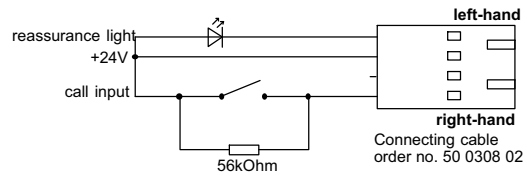**Note!** The complete installation of the system is described in the technical manual.

**Attention!** The printed circuit board includes electrostatic sensitive components. Avoid touching.

**Electromagnetic compatibility!** Due to insufficient suppression at fluorescent lamps in medical supply units faults in the call system may occur. These external interferences possibly can be avoided by installing suppressor elements (varistor circuits). Varistor circuits are available from the manufacturers of the medical supply units. Tunstall offers a proven overvoltage filter 230 V (order no. 70 0890 97).

Medical supply units may differ clearly in EMC behaviour. Even two supply units of the same type may differ, if the installation is different.

When laying call system cables in medical supply units the regulations of European standard ISO 11197 have to be followed.

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**Lit n° :**

ON	1	2	3	= Lit 1
ON	1	2	OFF	= Lit 2
ON	1	OFF	3	= Lit 3
ON	1	OFF	OFF	= Lit 4
OFF	1	2	3	= Lit 5
OFF	1	2	OFF	= Lit 6 *1

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**Setting coding switches**

**Bed no.:**

ON	1	2	3	= Bed 1
ON	1	2	OFF	= Bed 2
ON	1	OFF	3	= Bed 3
ON	1	OFF	OFF	= Bed 4
OFF	1	2	3	= Bed 5
OFF	1	2	OFF	= Bed 6 *1

**External call device:**

ON	5	6	= No external call device connected.
ON	OFF	6	= Normally open contact.
OFF	5	OFF	= Normally closed contact.

Coding switch 4 must always be set to OFF.

\*1 Attention: A diagnostic call can only be used, if a bed number is set. With bed number 6 no diagnostic call can be used.

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Gaine tête de lit  
Medical supply unit

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
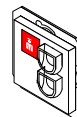
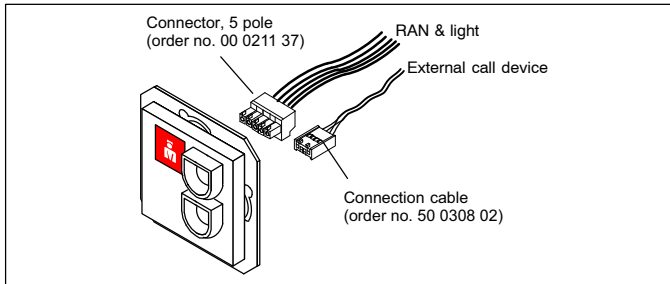
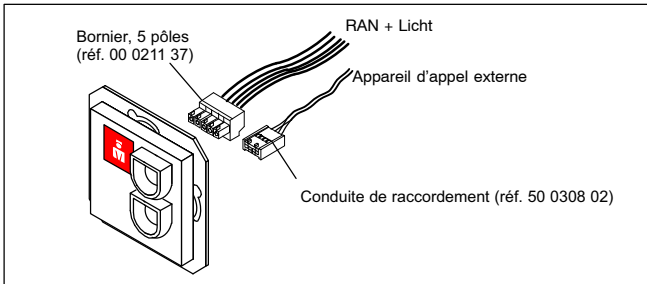
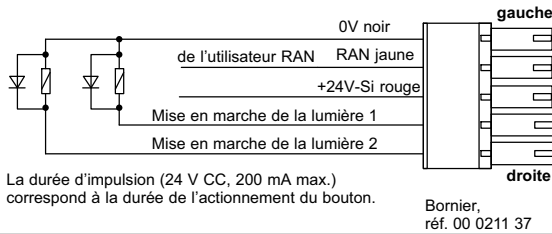
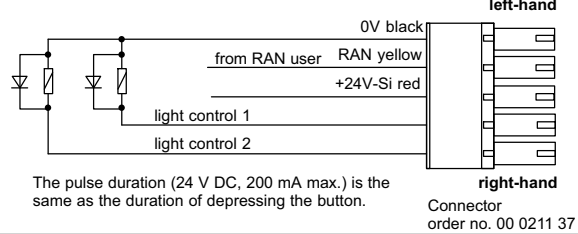
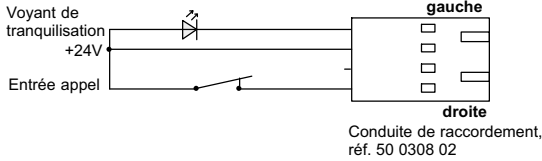
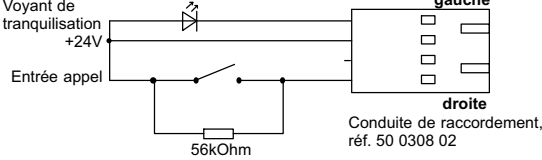
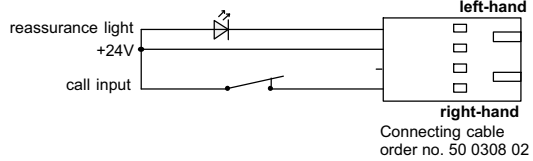
Connecting external call device as normally closed contact:

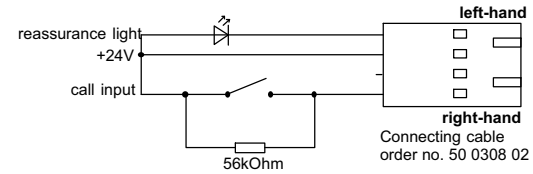Connecting external call device as normally open contact (e.g. Siemens Mediset):

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