RAN-Schnittstelle, Best.-Nr. 77 0840 00

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

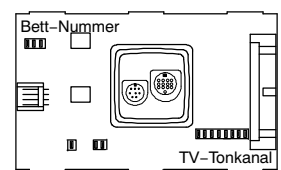
TV-Tonkanal:

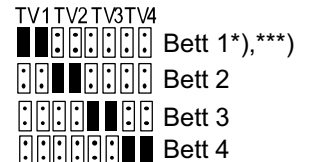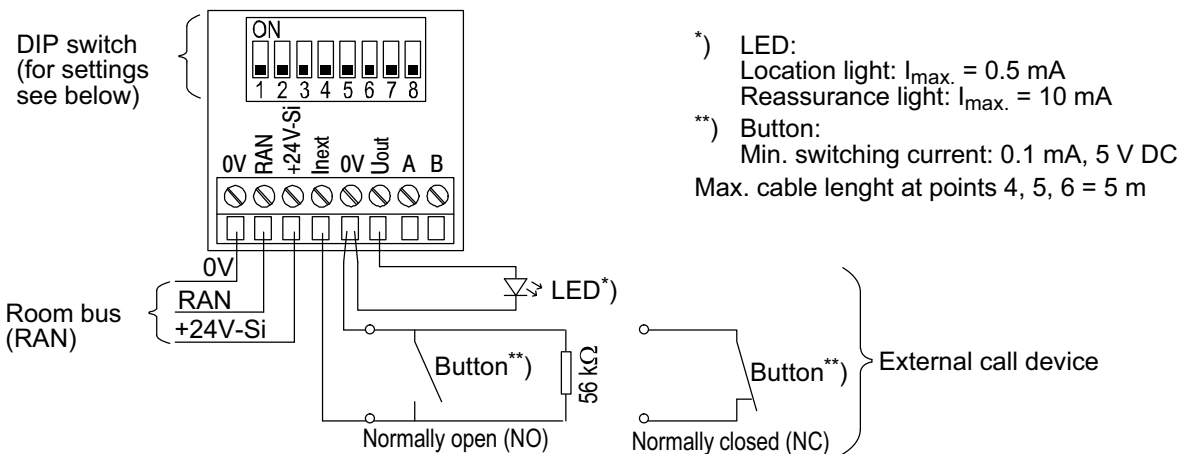B Application: An external call device raises the call type "call".
 Application: An external call device raises the call type "cardiac alarm".

*) LED:
 Location light: $I_{max.} = 0.5 \text{ mA}$
 Reassurance light: $I_{max.} = 10 \text{ mA}$
 **) Button:
 Min. switching current: 0.1 mA , 5 V DC
 Max. cable length at points 4, 5, 6 = 5 m

DIP switch 1–3:

Bed number:

- ON
- No bed (i.e. room call)
 - Bed 1
 - Bed 2
 - Bed 3
 - Bed 4
 - Bed 5
 - Bed 6

DIP switch 4:

Button contact type:

- ON
- Normally closed (NC)
 - Normally open (NO)

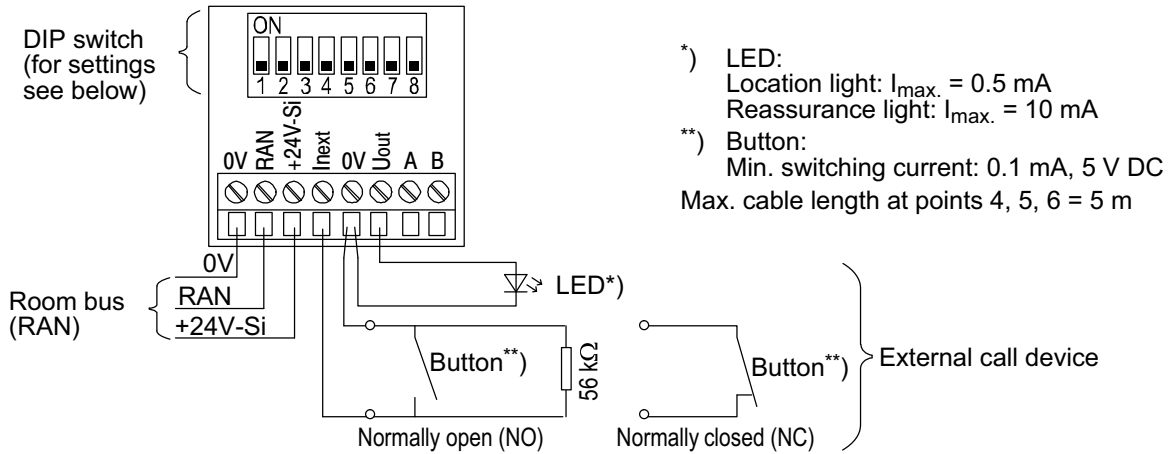
DIP switch 5–8:

Button function:


- ON
- Call type "call"
The call is cancelled in the nurse call system.
 - Call type "call"
The call is cancelled automatically, as soon as the call device is reset.
 - Call type "cardiac alarm"
The cardiac alarm is cancelled in the nurse call system.
 - Call type "cardiac alarm"
The cardiac alarm is cancelled automatically, as soon as the call device is reset.

The location light is on with a weak light. When a call is raised, it is on with a bright light as a reassurance light.


C Application: An external call device raises the call type "WC call".


DIP switch 1–3:
Channel number***):

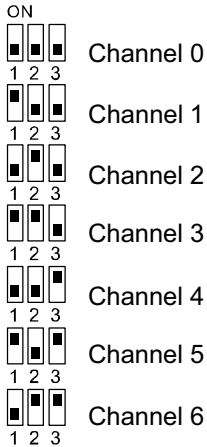DIP switch 4:
Button contact type:

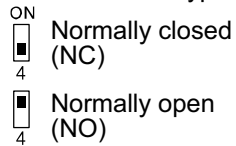
DIP switch 5–8:
Button function:

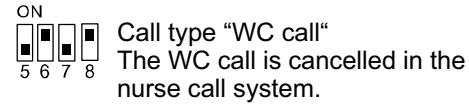
***) If functional units are requested, the call devices have to be assigned to different channels. Refer to your installation plan to find out, whether channel numbers have to be assigned.

The location light is on with a weak light. When a call is raised, it is on with a bright light as a reassurance light.

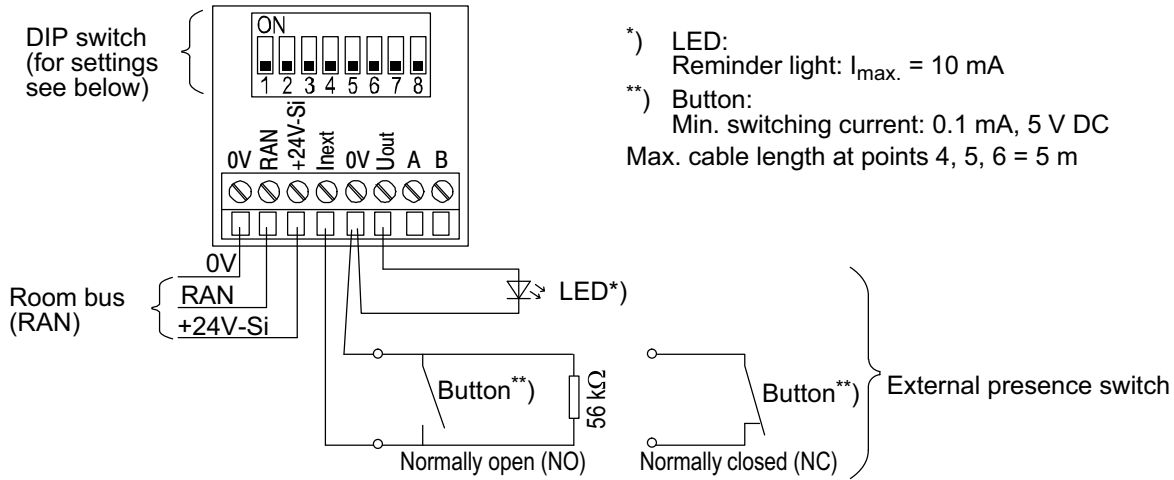
D Application: An external presence switch for switching staff presence 1 on and off.

DIP switch 1–3:
Channel No.***):

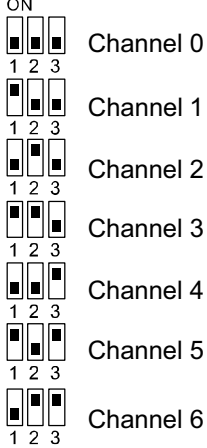DIP switch 4:
Button contact type:

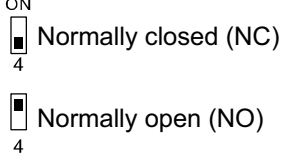
DIP switch 5–8:
Button function:


***) If functional units are requested, the presence switches have to be assigned to different channels. Refer to your installation plan to find out, whether channel numbers have to be assigned.

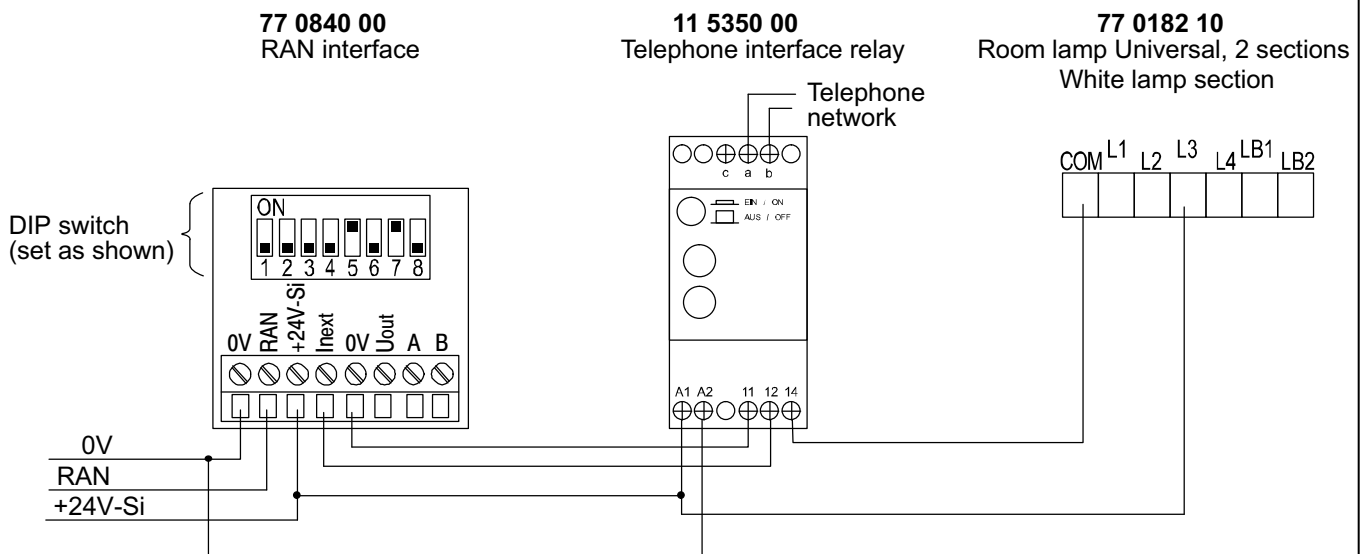
E Application: A telephone call raises the call type “telephone call“.

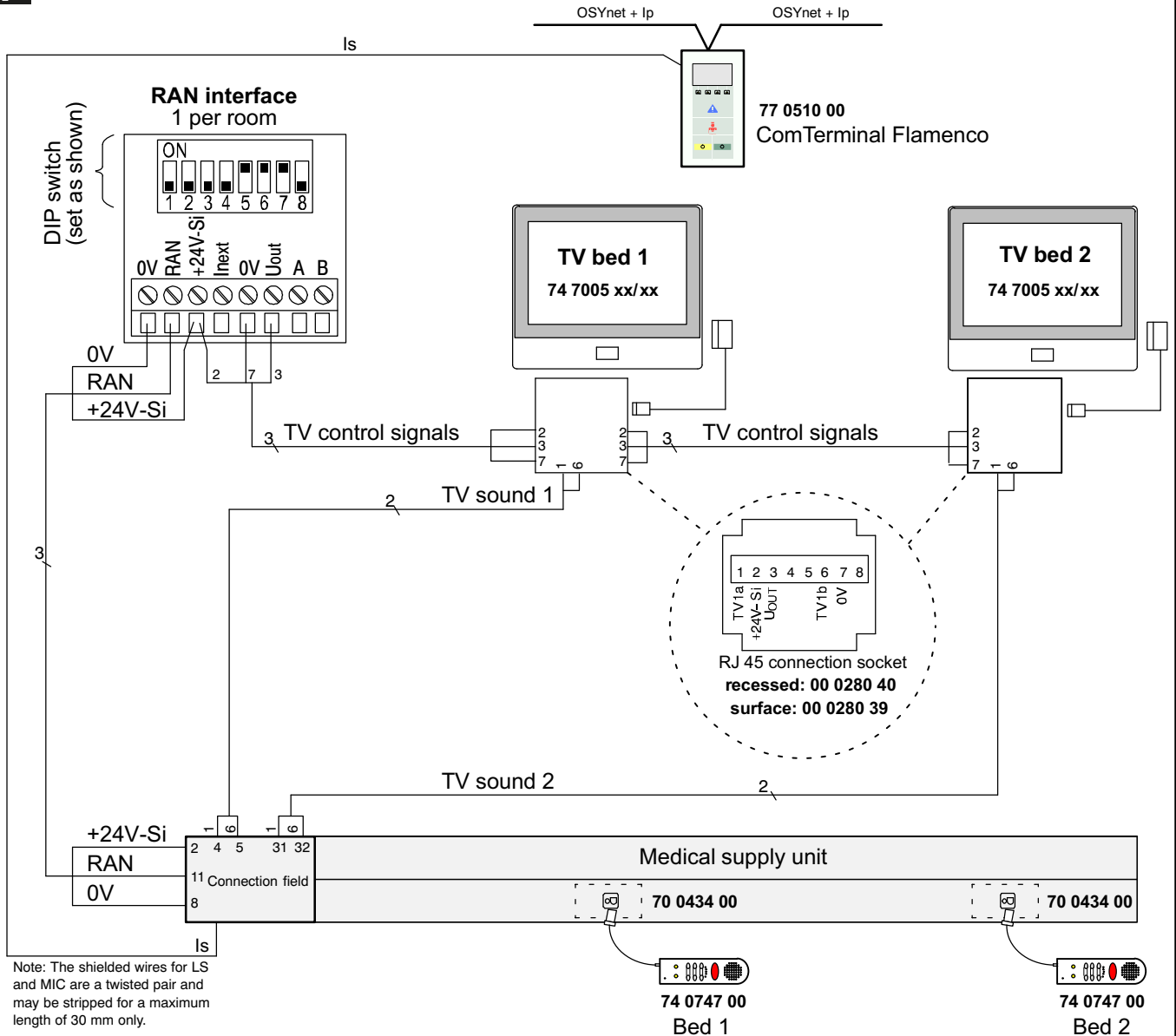F Application: Patient TV sets are controlled using a patient handset.

Connection field medical supply unit

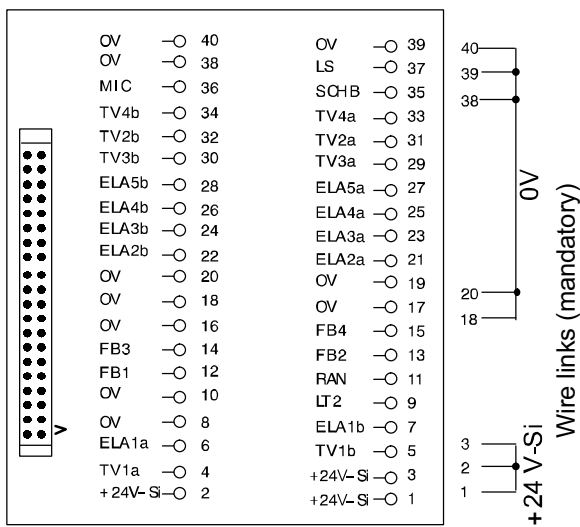
Terminal numbers as flat cable

Bed no. and TV sound channel

You have to set the bed number and the TV sound channel by setting jumpers on the connection socket (70 0434 00):

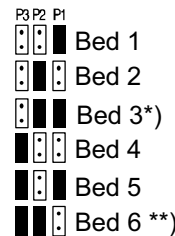
Bed number:


*) Factory setting.

**) When using bed 6 no diagnostic call can be used.

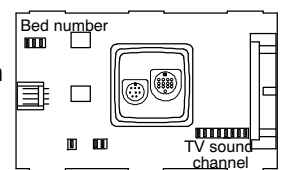***) If the same TV sound shall be transmitted to all beds, you have to set Bed 1 (TV1).

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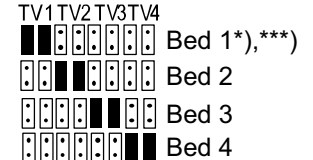RAN interface, order no. 77 0840 00

The RAN interface is intended for the connection of a third-party device to the room bus (RAN). Available applications:

- An external call device raises the call type "call".
- An external call device raises the call type "cardiac alarm".
- An external call device raises the call type "WC call".
- An external presence switch for staff presence 1.
- A telephone call raises the call type "telephone call".
- Patient TV sets are controlled using a patient handset.

NOTE! The complete installation of the system is described in the Technical Manual.

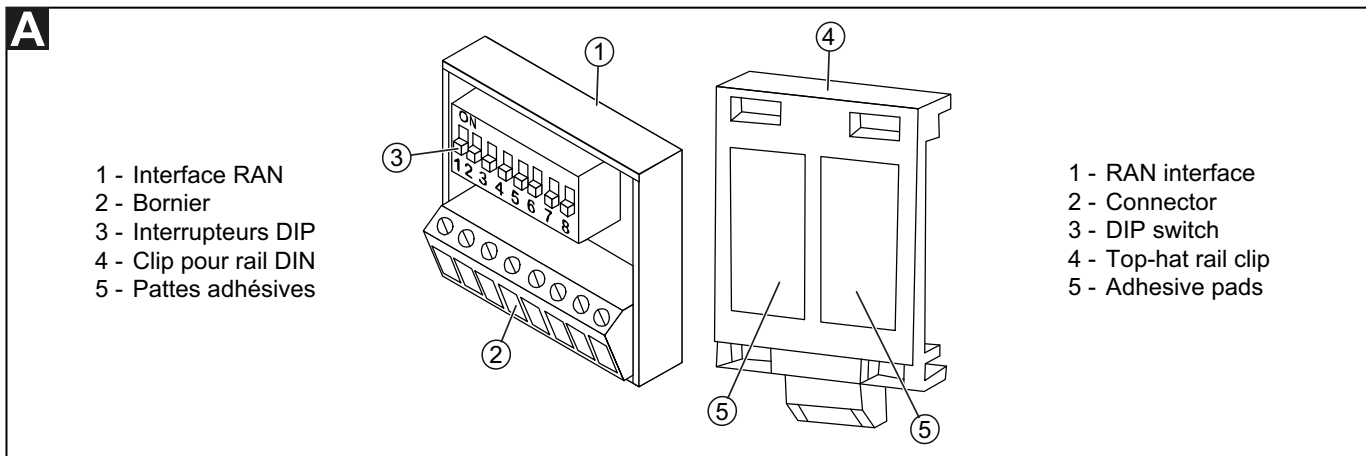
CAUTION! The printed circuit board includes electrostatic sensitive components. Avoid touching.

CAUTION! During installation the power supply should be switched off to ensure zero-potential status. Otherwise, the RAN interface could be damaged.

A Mounting

Fix the RAN interface [1] with the adhesive pads [5] to the top-hat rail clip [4]. Then clip it to the top-hat rail. See Fig. A.

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© Tunstall GmbH, Orkotten 66, 48291 Teigte, Germany, www.tunstall.de