Reinigung / Desinfektion

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Cleaning / Disinfection

The casing of the Patient handset is made of ABS plastic, which is suitable for moist wipe disinfection. This plastic has a good resistance to dilute organic and inorganic acids, alkaline substances, saline solutions as well as most animal and vegetable fats and oils.

On the other hand, concentrated mineral acids, aromatic hydrocarbons, hydrogen chloride, ester, ether, and ketone harm the plastic. Do not use cleaning fluid or disinfection fluid which contains any of these agents.

Additionally, you should not use harsh or abrasive cleaning material or scouring cream.

Non residue-free cleaning fluids collect in the spaces between the buttons and the casing.

It is essentially harmless to clean the Patient handset with a moist tissue using diluted or residue-free household cleaners or detergents.

Take care, that no fluid runs into the headset socket.

The Patient handset is not waterproof. Therefore don't dip the device into water.

Observe all valid hygiene regulations for the hospital.

Tunstall

PBK Hand

Bestell-Nr. 74 0747 00

Reinigung / Desinfektion

Patient handset

Order no. 74 0747 00

Cleaning / Disinfection