ComStation^{CT}, Tischgerät, Best.-Nr. 77 0601 00A

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ComStation^{CT}, desktop unit, order no. 77 0601 00A

Terminal with two-way speech communication, incl. red call button, blue code blue/cardiac alarm button, green presence button (staff 1), yellow presence button (staff 2), 4 function keys as soft keys, microphone and loudspeaker. Designed for desktop installation at the nurse station. Preferably intended for call handling and for speaking to rooms with activated staff presence. Raising of calls or cardiac alarms to summon assisting staff is possible.

Note! The complete installation of the system is described in the technical manual.

Attention! The printed circuit board includes electrostatic sensitive components. Avoid touching.

1. Start configuration menu

- Press the top and bottom function keys simultaneously (5 sec.) until "Configuration" is shown in the display. Symbol meanings:
 - ~ = Select menu item
 - + = Increase shown value
 - = Decrease shown value
 - X = Exit configuration menu (and store settings)

2. Entering of settings**Selecting the user language**

- Press "~" until "Language" is shown.
- Press "+" or "-" until the desired language is shown
D = German; GB = English; E = Spanish; F = French; NL = Dutch; I = Italian

Selecting the room type

- Press "~" until "Room type" is shown
- Press "+" or "-" until the room type "Staff room" is shown.
(The "Staff room" setting causes, that emergency calls are signalled as normal calls)

Setting of room address (01 - 63)

- Press "~" until "Address" is shown.
- Press "+" or "-" until the desired address is shown.

Setting of RAN number (00 - 30) (Room Area Network)

RAN number = Number of devices in the room (switches, connection sockets, room lamps, etc.) which are connected via RAN. (**No** per push switch, **no** patient handset, **no** connection socket ComStation)

- Press "~" until "RAN number" is shown.
- Press "+" or "-" until the desired RAN figure is shown.

3. Functional test "Test RAN"

Test if room devices are ready to operate and if they are correctly connected to the RAN. Check the correct setting of the RAN number.

- Press "~" until "Test CCS/Test RAN" is shown.
- Test start: Press "~" in the menu "Test CCS/Test RAN".
- The LEDs of all connected room devices must flash. (Non-flashing devices are wrongly connected or defective. Replace defective devices. Correct any wrong connection)
- Compare the number of connected room devices with the RAN number on the display [XX]. If the RAN number on the display is false, correct the setting after the RAN test as described above.
- End of test: Press "~" again.

4. Exit the configuration menu

When all settings are performed and all tests are finished, exit the configuration menu. During this procedure all settings are stored.

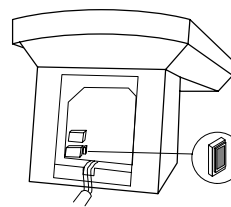
- Press "X" (= bottom function key).

Note: If for a period of three minutes no function key is pressed, the system will leave the configuration menu and the settings are stored.

5. Apply the room address

Integrated in the ComStation^{CT} is a printed circuit board (PCB), which must have the same room address as the address set in the configuration menu. This address has to be manually applied.

To do this, remove the cover from the rear side of the ComStation^{CT}.

1. Switch on staff presence at the ComStation^{CT}.
 2. Press the address button on the integrated PCB once.
 3. Switch off staff presence at the ComStation^{CT}.
- The address set in the configuration menu is now also stored in the integrated PCB.

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1. Start configuration menu

- Press the top and bottom function keys simultaneously (5 sec.) until "Configuration" is shown in the display. Symbol meanings:

- ~ = Select menu item
- + = Increase shown value
- = Decrease shown value
- X = Exit configuration menu (and store settings)

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RAN number = Number of devices in the room (switches, connection sockets, room lamps, etc.) which are connected via RAN. (**No** pear push switch, **no** patient handset, **no** connection socket ComStation)

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- Press "~" until "Test CCS/Test RAN" is shown.
- Test start: Press "-" in the menu "Test CCS/Test RAN".
- The LEDs of all connected room devices must flash. (Non-flashing devices are wrongly connected or defective. Replace defective devices. Correct any wrong connection)
- Compare the number of connected room devices with the RAN number on the display [XX]. If the RAN number on the display is false, correct the setting after the RAN test as described above.
- End of test: Press "-" again.

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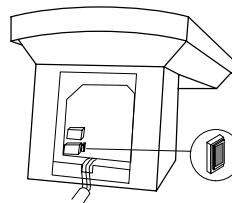
- Switch on staff presence at the ComStation^{CT}.
 - Press the address button on the integrated PCB once.
 - Switch off staff presence at the ComStation^{CT}.
- The address set in the configuration menu is now also stored in the integrated PCB.

address button
(mini switch)