ComTerminal Flamenco, Best.-Nr. 77 0510 00

ComTerminal Flamenco, order no. 77 0510 00

ComTerminal-F Flamenco, Best.-Nr. 77 0530 00

ComTerminal-F Flamenco, order no. 77 0530 00

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- | | |
|---|--------------------------------|
| 1 - * Four back box screws | 5 - * Back box |
| 2 - Pattress | 6 - Front panel |
| 3 - Two connections to the two loudspeakers | 7 - Flat cable |
| 4 - Two loudspeakers | 8 - Hinge |
| | 9 - Catch |
| | 10 - Security screw (optional) |

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After mounting the pattress, connect the connection cables to the connection field in the pattress [2] according to chapter "Connections".

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1. If a security screw [10] is fitted into the base of the ComTerminal, remove it.
2. Use a screwdriver to push the catch [9] upwards until the front panel [6] loosens.
3. Remove the front panel [6].
4. Disconnect the flat cable [7] from the pattress [2].
5. Lay down the front panel [6] protected. Caution: Electrostatic sensitive components! Caution: Glass!

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
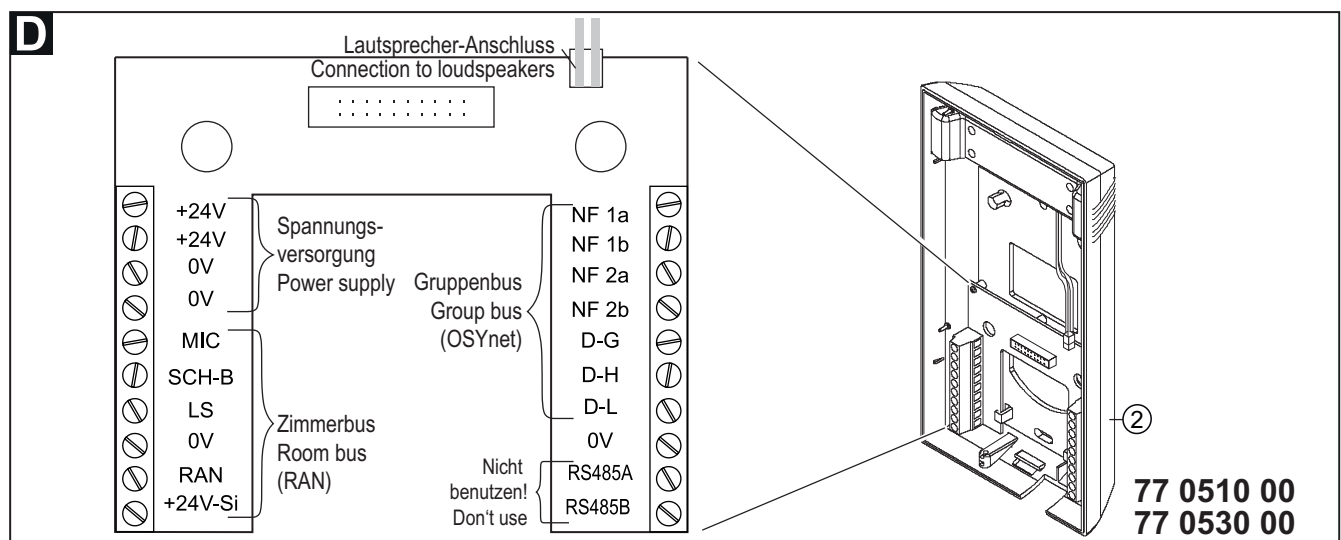
Observe the notes on the previous page!

<p>E</p> <p>Spannungsversorgung Kabel: NYM 2x2x2,5 mm²</p> <p>+24V _____ vom vorigen Zimmer oder zum nächsten Zimmer</p> <p>0V _____</p>	<p>Power supply Cable: NYM 2x2x2.5 mm²</p> <p>+24V _____ from previous room or to next room</p> <p>0V _____</p>
<p>Zimmerbus (RAN) mit Sprechen Kabel: 2 x IY(ST)Y 2x2x0,8 (max. 2 Adern pro Klemmpunkt)</p> <p>+24V-Si _____ rt RAN _____ ge 0V _____ sw ws</p> <p>zur Steckvorrichtung Kombi oder Kombi Kanal</p> <p>LS _____ ge MIC _____ ws SCH B _____ sw rt</p>	<p>Room bus (RAN) with speech Cable: 2 x IY(ST)Y 2x2x0.8 (max. 2 wires per connection point)</p> <p>+24V-Si _____ red RAN _____ yellow 0V _____ black white</p> <p>to connection socket combi or combi, bedhead unit</p> <p>LS _____ yellow MIC _____ white SCH B _____ black red</p>
<p>Zimmerbus (RAN) mit Sprechen Kabel: CCS32 (max. 2 Adern pro Klemmpunkt)</p> <p>+24V-Si _____ br 0V _____ sw RAN _____ gr LS _____ ws MIC _____ bl SCH B _____</p> <p>zur Steckvorrichtung Kombi oder Kombi Kanal</p>	<p>Room bus (RAN) with speech Cable: CCS32 (max. 2 wires per connection point)</p> <p>+24V-Si _____ brown 0V _____ black RAN _____ grey LS _____ white MIC _____ blue SCH B _____ shield</p> <p>to connection socket combi or combi, bedhead unit</p>
<p>Zimmerbus (RAN) ohne Sprechen Kabel: IY(ST)Y 2x2x0,8 (max. 4 Adern pro Klemmpunkt)</p> <p>+24V-Si _____ rt RAN _____ ge 0V _____ sw ws</p> <p>zum RAN-Teilnehmer</p>	<p>Room bus (RAN) without speech Cable: IY(ST)Y 2x2x0.8 (max. 4 wires per connection point)</p> <p>+24V-Si _____ red RAN _____ yellow 0V _____ black white</p> <p>to RAN user</p>
<p>Gruppenbus (OSYnet) Kabel: CAT7 (22 AWG) oder CAT5 (23 AWG)</p> <p>DL _____ ws DH _____ bl gn DG _____ ws NF 1a _____ br NF 1b _____ ws NF 2a _____ or NF 2b _____ ws</p> <p>vom vorigen Zimmer oder zum nächsten Zimmer</p>	<p>Group bus (OSYnet) Cable: CAT7 (22 AWG) or CAT5 (23 AWG)</p> <p>DL _____ white DH _____ blue green DG _____ white NF 1a _____ brown NF 1b _____ white NF 2a _____ orange NF 2b _____ white</p> <p>from previous room or to next room</p>
<p>Gruppenbus (OSYnet) Kabel: IY(ST)Y 4x2x0,8</p> <p>DL _____ rt DH _____ bl gn DG _____ ws NF 1a _____ br NF 1b _____ ws NF 2a _____ ge NF 2b _____ ws</p> <p>vom vorigen Zimmer oder zum nächsten Zimmer</p>	<p>Group bus (OSYnet) Cable: IY(ST)Y 4x2x0,8</p> <p>DL _____ red DH _____ blue green DG _____ white NF 1a _____ brown NF 1b _____ white NF 2a _____ yellow NF 2b _____ white</p> <p>from previous room or to next room</p>

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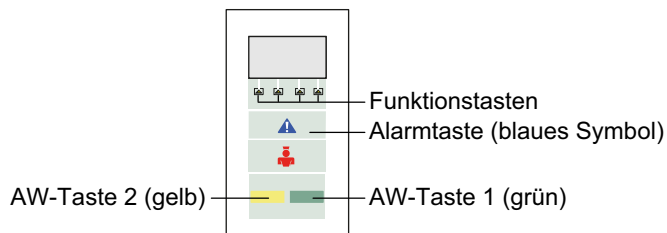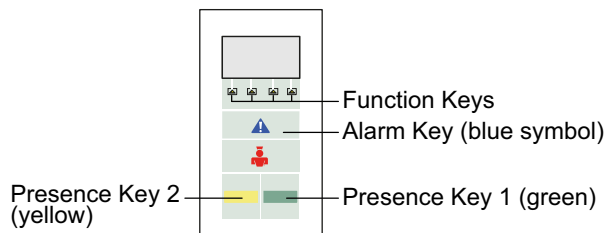
Symbols in the configuration menu

●	Circle key	Set the marked menu item.
▲	Upwards arrow key	Scroll upwards through the list.
▼	Downwards arrow key	Scroll downwards through the list.
⏠	Home key	Cancel, without storing.

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Menu items

- Language: Selecting the user language, e.g. English.
- Room type: Selecting the room type, e.g. patient room.
- RAN > RAN number: Setting of number of room devices (= RAN number).
- RAN > Test RAN: Test if room devices are ready to operate and are correctly connected to the RAN.
- RAN > Status: Check whether the stored RAN number is equal to the number of operational room devices.
- Address: Setting of room address.
- Cleaning time: Setting the time (seconds) how long it takes to clean (wipe) the front of the ComTerminal.
- Contrast: Function use only for Tunstall technicians.
- Key sound: Switching the key sound of the ComTerminal on or off. "Key sound On" (factory setting) is recommended.
- Info: Displaying the ComTerminal's software revision.
- Audio Test: Only for Tunstall technicians.
- Key Function: Setting the function of the Alarm key and the Presence Keys on the ComTerminal.
- Reset: Function use only for Tunstall technicians. (Restart the ComTerminal).

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Selecting the room type

1. Mark "Room type" using the arrow keys. Then press the circle key.
2. Mark the desired room type using the arrow keys:
 - ♦ Patient room
 - ♦ Patient room with cancel key for WC call at ComTerminal
 - ♦ Staff room (emergency calls are signalled as normal calls)
 - ♦ Children's room (no key to switch the privacy function at ComTerminal, Privacy OFF)
 - ♦ Connection terminal (keys and display at ComTerminal non-operational)
3. Press the circle key to set the selection.

Setting of RAN number (0 – 30) (Room Area Network)

RAN number = Number of devices in the room (switches, connection sockets, room lamps, etc.) that are connected via RAN. (**No** pear push switch).

1. Mark "RAN" using the arrow keys. Then press the circle key.
2. Mark "RAN number" using the arrow keys. Then press the circle key.
3. Mark the desired RAN number using the arrow keys.
4. Press the circle key to set the selection.

Setting of room address (0 – 110)

1. Mark "Address" using the arrow keys. Then press the circle key.
2. Mark the desired room address using the arrow keys.
3. Press the circle key to set the selection.

3. Entering of optional settings

The remaining menu items provide optional settings. Set these if required. Examples:

Key Function

CAUTION! Switched off keys are not available for nursing staff!

Alarm Key

With the factory settings the alarm key on the ComTerminal is operational and is initiated by pressing it.

NOTE! Die Alarm Key is only active while the staff presence is switched on in the room.

If there is the risk, that the alarm button is initiated inadvertently, it might make sense to set a delay time (2 or 3 seconds). In that case the alarm will be initiated after the alarm key has been pressed and then hold for 2 or 3 seconds.

CAUTION! Setting a delay time does not comply with the German standard DIN VDE 0834. A delay time may only be set, where compliance with the DIN VDE 0834 is not required.

In well-founded exceptional cases you can switch of the alarm key function.

WARNING! It is not allowed to switch off the function of the alarm key of the ComTerminal 77053000, because it is not visible to the user that the alarm button has no function.

It is allowed to switch off the function of the alarm key of the ComTerminal 77051000, because no alarm key is visible any longer, if the function of the alarm key is switched off.

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Presence Key 1 / Presence Key 2

With the factory settings the presence keys on the ComTerminal are operational.

In well-founded exceptional cases you can switch of the key function for these keys.

4. Checking the room bus RAN**Status**

Check whether the set RAN number (refer to section “Setting of RAN number” of chapter “2. Entering of necessary settings”) equals to the number of operational room devices.

1. Mark “RAN” using the arrow keys. Then press the circle key.
2. Mark “Status” using the arrow keys. Then press the circle key.

Display: **OK**: The set RAN number is equal to the number of operational room devices.

Display: **Error**: The set RAN number is not equal to the number of operational room devices.

3. Press the home key to end. In case of an **error** perform a “Test RAN”.

Test RAN

Test if room devices are ready to operate and if they are correctly connected to the RAN.

Check the correct setting of the RAN number.

1. Mark “RAN” using the arrow keys. Then press the circle key.
2. Test start: Mark “Test RAN” using the arrow keys. Then press the circle key.

The LEDs of all connected room devices must flash. Non-flashing devices are wrongly connected or defective. Replace defective devices. Correct any wrong connection.

3. Compare the number of flashing room devices with the set RAN number. If the set RAN number is wrong, correct the RAN number setting after the RAN test as described in section “Setting of RAN number” of chapter “2. Entering of necessary settings”.
4. End of test: Press the home key.

5. Exit the configuration menu

When all settings are made and all tests are completed, you have to exit the configuration menu:

- Press home key several times until normal operation display appears.

NOTE! If for a period of three minutes no function key is pressed, the system will leave the configuration menu.

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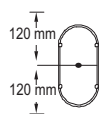
Communication terminal with two-way speech communication. Red call button, blue alarm button, green presence button (staff 1), yellow presence button (staff 2) and 4 function keys as soft keys. Microphone and loudspeakers.

Control and monitoring of all room functions in compliance with DIN VDE 0834. Connection to the group bus (OSYnet) and the room bus (RAN).

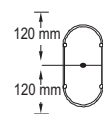
NOTE! The complete installation of the system is described in the Technical Manual.

CAUTION! The printed circuit board includes electrostatic sensitive components. Avoid touching.

Glass – fragile! The operating front cover of the ComTerminal 770510000 is made of glass. Handle with care!


Back box installation

When installing the back box consider the space required for the ComTerminal. Measuring from the centre of the box, the available space must be at least 120 mm above and below.

A Mounting the pattress

1. Insert the connection cables through the outlet in the pattress [2].
2. Fix the pattress [2] using the back box screws [1] to the back box [5].

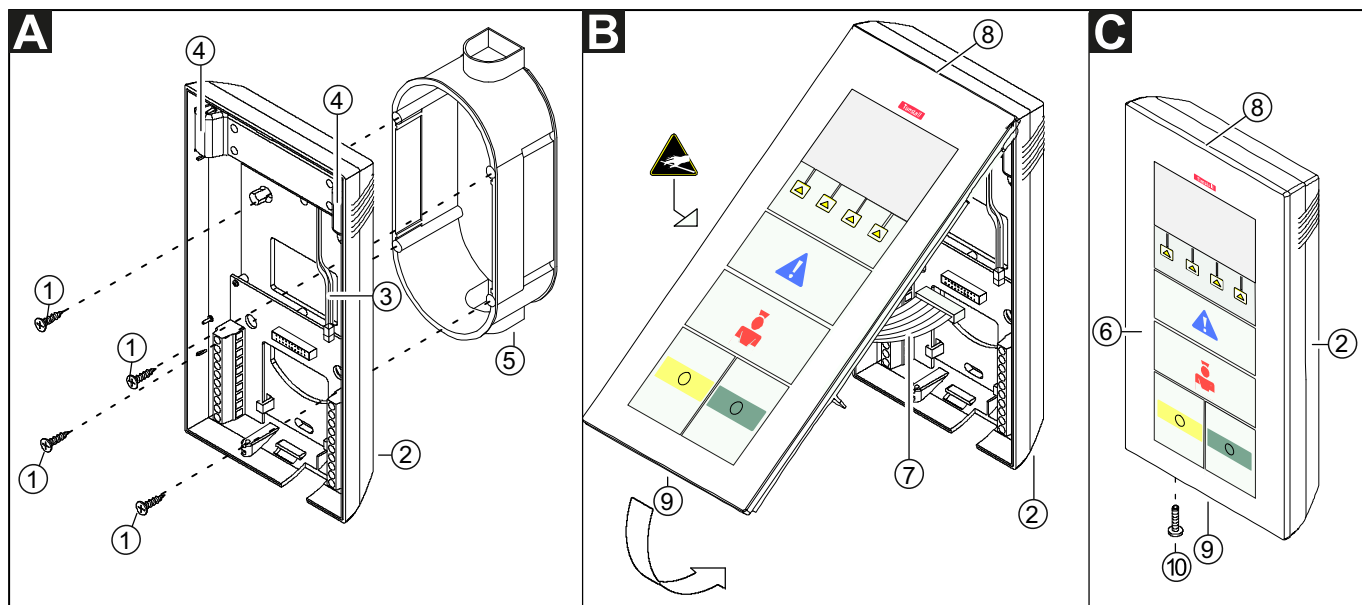NOTE! If mounting on a back box is not possible, the ComTerminal may be fixed to the wall with dowels and screws using the same holes in the pattress.

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- | | |
|---|--------------------------------|
| 1 - * Four back box screws | 6 - Front panel |
| 2 - Pattress | 7 - Flat cable |
| 3 - Two connections to the two loudspeakers | 8 - Hinge |
| 4 - Two loudspeakers | 9 - Catch |
| 5 - * Back box | 10 - Security screw (optional) |

* Not included with ComTerminal delivery

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EN - Installation Instructions

After mounting the pattress [2], connect the connection cables to the connection field in the pattress [2] according to chapter "Connections".

B Mounting the front panel

1. Plug the flat cable [7] of the front panel [6] into the appropriate socket on the connection field in the pattress [2].
2. Place front panel [6] onto the top of the pattress [2] creating a hinge [8].
3. Press the front panel downwards onto the pattress with low pressing power (risk of glass breakage for ComTerminal 77051000!) until the catch [9] engages. Make sure that the flat cable [7] is not pinched.
4. For improved security the security screw [10] can be fitted into the hole below the yellow presence button.

Finally, the ComTerminal has to be configured according to the pages "Configuration Instructions".

C Dismantling

1. If a security screw [10] is fitted into the base of the ComTerminal, remove it.
2. Use a screwdriver to push the catch [9] upwards until the front panel [6] looses.
3. Remove the front panel [6].
4. Disconnect the flat cable [7] from the pattress [2].
5. Lay down the front panel [6] protected. Caution: Electrostatic sensitive components! Caution: Glass!

D Connections

IMPORTANT NOTES

- All cables for speech transmission (shielded cables) may be only be stripped to a maximum of 30 mm to maintain the shielding function.
- Insulate the shield wire (SCH-B) to prevent short circuiting.
- Tighten the screws from unused connectors to prevent screws from vibrating when the speakers are active.

Connection capacity	0.20 – 2.50 mm ²
Screwdriver blade	0.6 x 3.5 mm
Insulation strip length	6 mm

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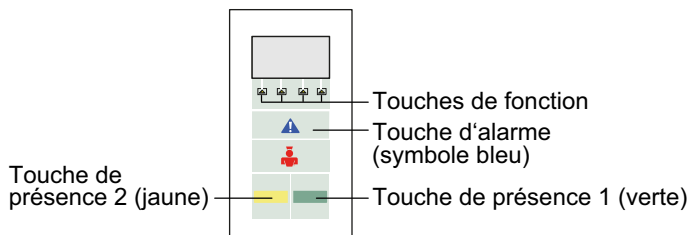
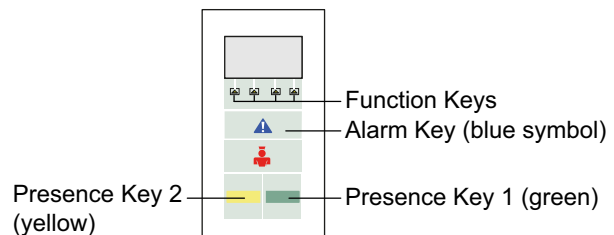
1. Start configuration menu

- Press the left and right function keys simultaneously (3 sec.) until the configuration menu is shown in the display.

Symbols in the configuration menu

●	Circle key	Set the marked menu item.
▲	Upwards arrow key	Scroll upwards through the list.
▼	Downwards arrow key	Scroll downwards through the list.
⌂	Home key	Cancel, without storing.

Menu items

Language:	Selecting the user language, e.g. English.
Room type:	Selecting the room type, e.g. patient room.
RAN > RAN number:	Setting of number of room devices (= RAN number).
RAN > Test RAN:	Test if room devices are ready to operate and are correctly connected to the RAN.
RAN > Status:	Check whether the stored RAN number is equal to the number of operational room devices.
Address:	Setting of room address.
Cleaning time:	Setting the time (seconds) how long it takes to clean (wipe) the front of the ComTerminal.
Contrast:	Function use only for Tunstall technicians.
Key sound:	Switching the key sound of the ComTerminal on or off. "Key sound On" (factory setting) is recommended.
Info:	Displaying the ComTerminal's software revision.
Audio Test:	Only for Tunstall technicians.
Key Function:	Setting the function of the Alarm key and the Presence Keys on the ComTerminal.
Reset:	Function use only for Tunstall technicians. (Restart the ComTerminal).

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4. Checking the room bus RAN

Status

Check whether the set RAN number (refer to section "Setting of RAN number" of chapter "2. Entering of necessary settings") equals to the number of operational room devices.

1. Mark "RAN" using the arrow keys. Then press the circle key.
2. Mark "Status" using the arrow keys. Then press the circle key.

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3. Press the home key to end. In case of an **error** perform a "Test RAN".

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3. Compare the number of flashing room devices with the set RAN number. If the set RAN number is wrong, correct the RAN number setting after the RAN test as described in section "Setting of RAN number" of chapter "2. Entering of necessary settings".
4. End of test: Press the home key.

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