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**Diagnostic connection cable,  
order no. 19 0702 90**

3 m connection cable for connecting a medical electrical device to the nurse call system via a call switch with 2 connection sockets, order no. 19 0704 20.

**WARNING!** The transmission of alarm conditions of medical electrical equipment (e.g. monitoring devices) to the nurse call system serves only as supporting, additional information. This is a distributed information system.

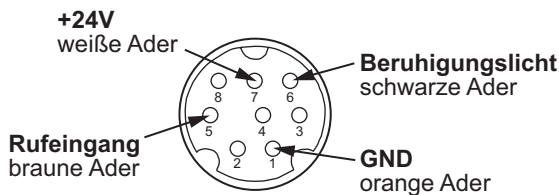
- Medical staff must **not** rely on the nurse call system displays.
- Medical staff must not leave the hearing and visual range of the medical monitoring equipment.
- Due diligence for the operation of such medical electrical equipment remains unaffected in case of the connection to the nurse call system.

The diagnostic connection cable is therefore marked with the following warning notice: „WARNING! No secure, reliable connection acc. to IEC 60601-1-8! There is a danger of not recognising alarm situations.“


**Note!** The complete installation of the system is described in the Technical Manual.

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
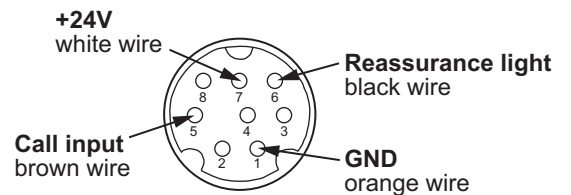
**Front view of a socket  
of the call switch with 2 connection sockets (19 0704 20)**

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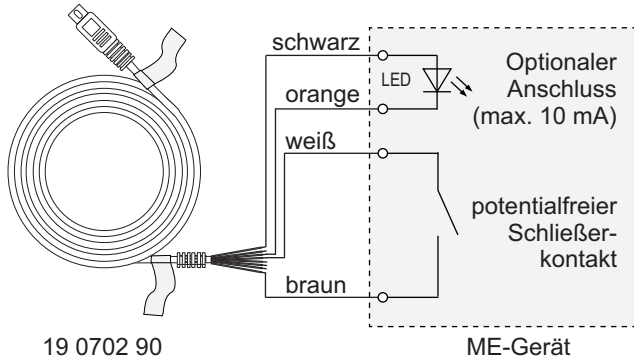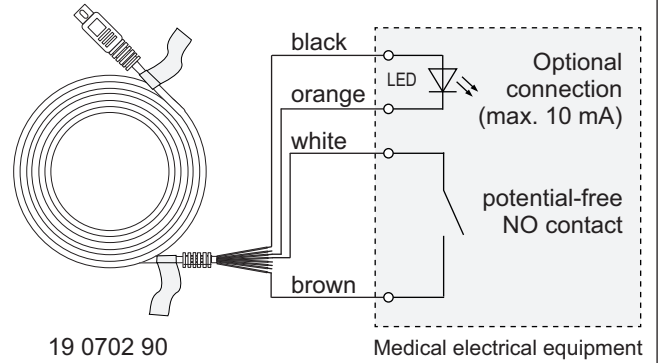The free wire ends (30 mm) are tin-plated.

**CAUTION!** The non used wire ends must be insulated.

**Configure the call type „Diagnostic call“**

In order for the medical electrical equipment to trigger a diagnostic call in the nurse call system, the room control input to which the call switch with 2 connection sockets is connected must be configured as follows:

- Call type: „Diagnostic“, „Diagnostic bed x“.
- NO contact
- Static call

Details can be found in the installation instructions for the room control unit.