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**Diagnostic connection cable,  
order no. 70 0812 10**

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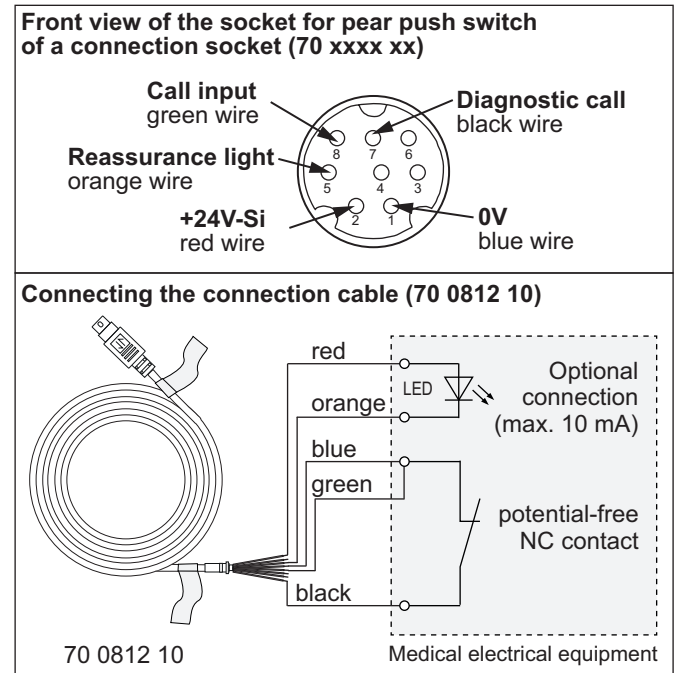
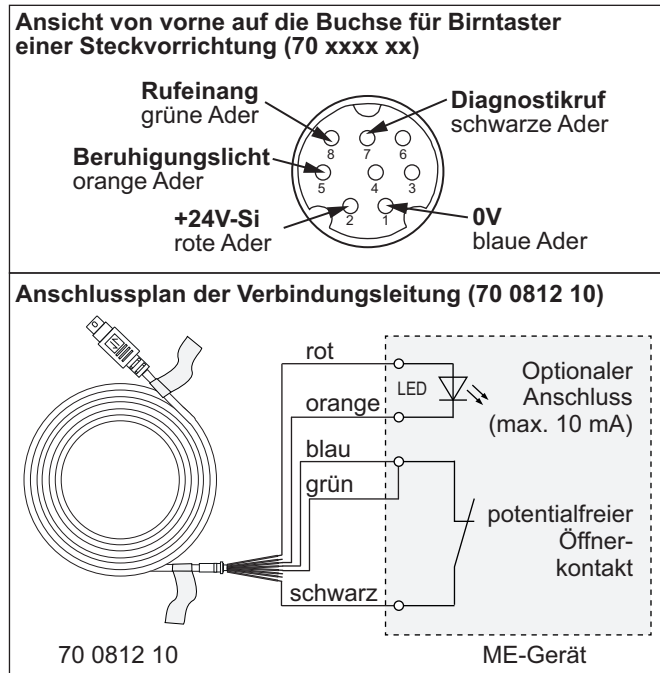
The diagnostic connection cable is therefore marked with the following warning notice: „WARNING! No secure, reliable connection acc. to IEC 60601-1-8! There is a danger of not recognising alarm situations.“

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**CAUTION!** The non used wire ends must be insulated.

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**NOTE!** The complete installation of the nurse call system is described in the Technical Manual.

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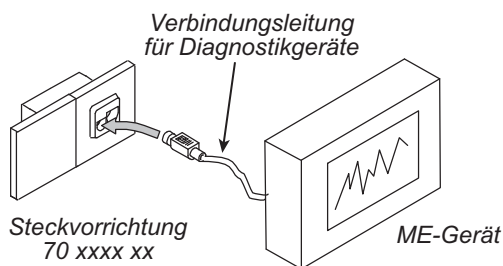
## Diagnostic connection cable, order no. 70 0812 10

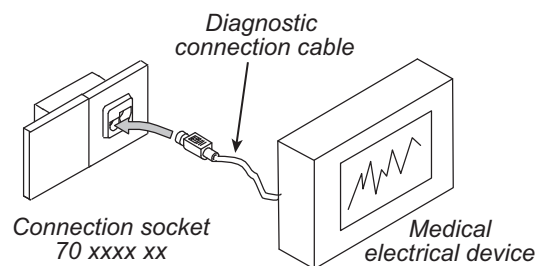
2 m connection cable for connecting a medical electrical device to the nurse call system via a connection socket, order no. 70 xxxx xx, in order to trigger a diagnostic call in the nurse call system for alarms of the medical electrical equipment.

**WARNING!** The transmission of alarm conditions of medical electrical equipment (e.g. monitoring devices) to the nurse call system serves only as supporting, additional information. This is a distributed information system.

- Medical staff must **not** rely on the nurse call system displays.
- Medical staff must not leave the hearing and visual range of the medical monitoring equipment.
- Due diligence for the operation of such medical electrical equipment remains unaffected in case of the connection to the nurse call system.

### Prepare the diagnostic call

1. Plug the diagnostic connection cable into the connection socket 70 xxxx xx (socket for pear push switch).

### Diagnostic call is raised

If a defined threshold value is exceeded on the medical electrical device, a diagnostic call is triggered in the nurse call system:

- The room lamp is flashing red. A diagnostic call is displayed on the call handling console.
- In case the diagnostic call is not answered in a pre-set time, it is forwarded to all rooms where the presence button is activated.
- If the presence button is activated at the call location, not a fresh call but only an answered call is raised, i.e. the call will not be forwarded.

### Cancel the diagnostic call

1. Enter the call location and activate the presence button.
2. Reset the medical electrical device.
3. De-activate the presence button.

The call is cancelled. The call is no longer displayed.