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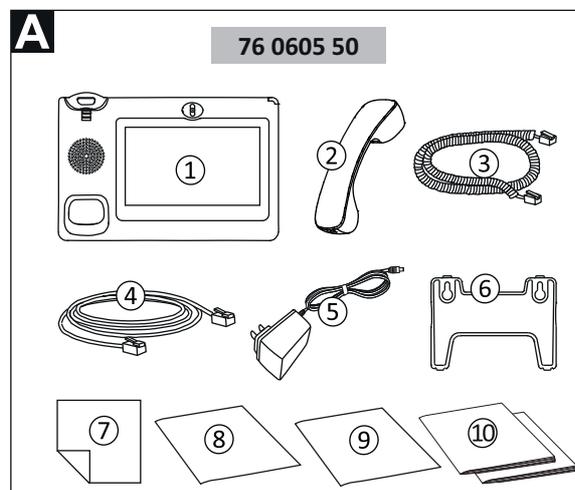
**ComStation^{IP}, order no. 76 0605 50****1. Intended use**

The ComStation^{IP} is intended for plug-in connection to the IP network of a Flamenco^{IP+} nurse call system.

The ComStation^{IP} is a ward console with the following functions:

- Answering and processing of calls
- Announcements
- Switching ward couplings on and off
- Switching shifts on and off
- Optional free or discreet speech

The ComStation^{IP} is based on an IP desk telephone from manufacturer GRANDSTREAM with adapted Android operating system. The nurse call functionality is pre-installed as an application. All other functions are inactive (kiosk mode).

2. Important documents**Flamenco system family, Technical Manual (00 8813 97)**

The complete installation of the system is described in the technical manual. The manual can be downloaded from the www.tunstall.de website.

App help

The ComStation^{IP} app has an integrated help, which describes the operation of the device in detail.

User guide (00 8813 85)

A detailed user guide can be downloaded as PDF from the www.tunstall.de website.

Quick reference guide (00 8813 87)

A quick reference guide is included with the ComStation^{IP} and can be downloaded from www.tunstall.de.

Documentation by GRANDSTREAM

The ComStation^{IP} is supplied with documents from the device manufacturer GRANDSTREAM. Keep these. These documents are not intended for nursing staff.

3. Scope of supply (→ fig. A)

- [1] Main case
- [2] Handset
- [3] Handset cord
- [4] Ethernet cable
- [5] 12 V power adaptor
- [6] Wall mount
- [7] Screen cleaning cloth
- [8] Installation instructions (this document)
- [9] Quick reference guide
- [10] Documents by GRANDSTREAM

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4. Misuse

The ComStation^{IP} is based on an IP desk telephone from manufacturer GRANDSTREAM with adapted Android operating system. The nurse call functionality is pre-installed as an application. All other functions are inactive (kiosk mode).

Take note:

- The IP desk telephone may only be used as ComStation^{IP}. No other applications may be installed or used.
- The IP desk telephone must not be connected to the Internet. Only updates from Tunstall may be installed on the IP desk telephone.
- The Android operating system must not be reset to factory settings. This would result in the loss of changes to the operating system that can only be made by Tunstall.

5. Power supply

5.1 Power supply options

The device can optionally be supplied with power as follows:

- with the supplied 12 V power adaptor* (input: 100 – 240 V AC, 50 – 60 Hz; output 12 V DC, 1.5 A).
- in a LAN with PoE/PoE+ (PoE 802.3af Class 3, PoE+ 802.3at, Class 4).

*) Only use the 12 V DC power adaptor included in the scope of delivery of the device. Use of another power adapter may damage the unit.

5.2 Power source for safety purposes

The German standard DIN VDE 0834 stipulates that in the event of a failure of the general power supply, the nurse call system must be supplied from a power source for safety purposes. This means that the 230 AC socket respectively the PoE/PoE+ network switch or injector must be connected to an UPS/auxiliary power supply.

6. Location for the device

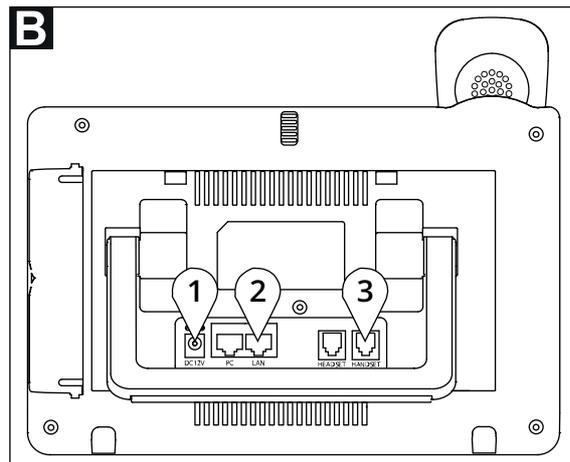
- Minimum distance of 20 cm between the device and the body of the user.
- The area in front of the microphone (front right) should be kept clear. The optimum distance for hands-free talking is 50 cm.
- Do not place the device in a room with high dust generation, as this can considerably reduce the service life of the device.
- No direct sunlight or other heat radiation, as this can damage the electronic components and the plastic housing.
- Not in environments where steam is generated.

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7.1 Ports

- [1] Power jack for 12 V DC power adaptor (only required if the device is not powered via the Ethernet cable (PoE))
- [2] LAN port
- [3] RJ9 handset connector port

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7.2 Proceed as follows

i A **prerequisite** for using the ComStation^{IP} is that the nurse call system has been configured for the use using the SystemOrganizer configuration software.

1. Connect the handset cord to the handset and to the port for handset cord [3].
2. Connect the Ethernet cable to the LAN port [2] and to the RJ45 connection box in the room. The RJ45 connection box must be connected to the nurse call system according to the Technical Manual.

CAUTION! Make sure that the latch of the RJ plugs engages so that the plugs cannot slip out.

3. If the device is not powered by PoE or PoE+ (PoE network switch), connect the 12 V DC power adaptor to the power jack [1] and to a 230 V socket.

CAUTION! DO NOT switch off the device during system start-up or when updating the firmware. Doing so may damage the firmware.

- ✓ The device is starting up. The ComStation^{IP} app login screen appears automatically.
- 4. Tap “LOG IN”.
- ✓ You will be logged in. The Home screen of the ComStation^{IP} app appears. The ComStation^{IP} is operating.

CAUTION! You have to secure all plugs (Ethernet cable, 12 V power adaptor) against unintentional removal. It is recommended to label the RJ45 connection box and the 230 V socket with a warning: “DANGER! Do not pull out the plug! Device is part of the nurse call system”.

For information on how to operate the ComStation^{IP} or change default settings, please refer to the App Help. To open the help, tap on the question mark in the home screen.

Functional checks must be carried out as described in the technical manual, before the ComStation^{IP} can be used.

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8. Desktop installation / wall mounting (→ fig. C, D)

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8.1 Desktop installation

The device has an integrated stand [2]. To use it, fold it out of the back of the device and adjust it to the desired angle. Ensure that the device is in a secure position.

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8.2 Wall mounting

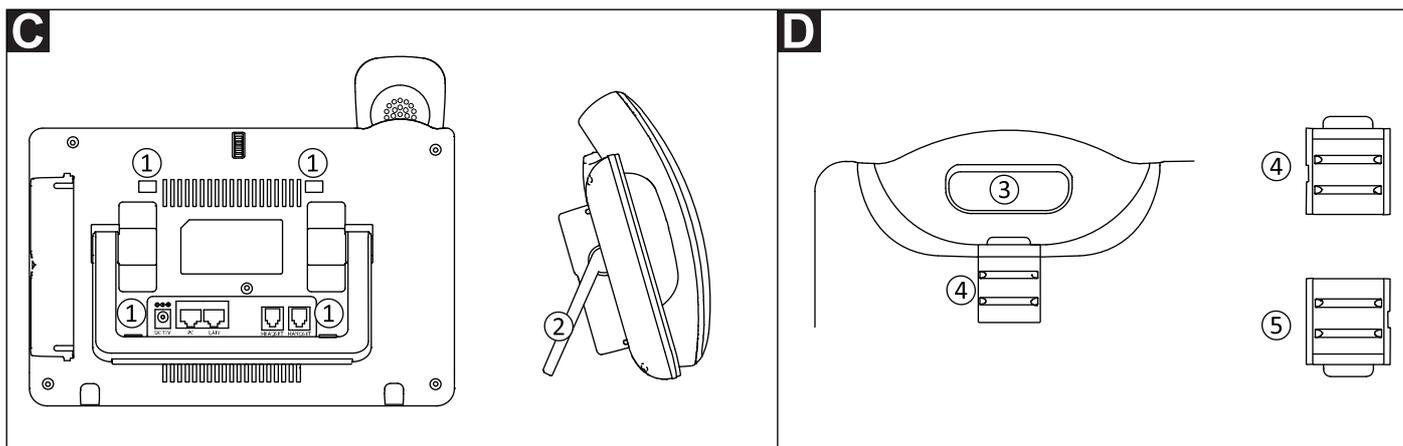
1. Attach the wall mount to the two mounting slots [1] on the back of the device.
2. Attach the device to the wall via the two key holes in the wall mount.
3. Pull out the tab from the handset cradle [3].
4. Rotate the tab and plug it back into the slot with the extension up [4].

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- [1] Four mounting slots for fixing the wall mount
 [2] Built-in stand for desktop installation
 [3] Handset cradle
 [4] Tab in the position for wall mounting (tab with extension up)
 [5] Tab in the position for desktop installation (tab with extension down)

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9. Technical data

Dimensions (HxWxD)	84 x 252 x 211 mm
Weight	1080 g
Power consumption	– via PoE: approx. 11 W – via Power adaptor: approx. 9 W
Ambient temperature: - Transport, storage - Operation	-10 °C to +60 °C 0 °C to +40 °C
Relative humidity	10 % – 90 % (non condensing)