Zugtaster, Bestell-Nr. 19 0707 20x

- 19 0707 20J: inkl. Rahmen Jung AS 500
 19 0707 20B: inkl. Rahmen Berker S.1
 19 0707 20G: inkl. Rahmen Gira Standard 55 matt
 19 0707 20GG: inkl. Rahmen Gira Standard 55 glänzend
 19 0707 20M: inkl. Rahmen Merten System M-SMART

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The pull cord switch is intended for connection to one of the following room terminals:

- Terminal, order no.: 19 0700 x0, 10 0700 x0
- Display module, order no.: 19 0700 80, 10 0700 8x
- RAB, order no. 19 0701 00, 10 0701 x0
- Control module RL, order no. 19 0701 10

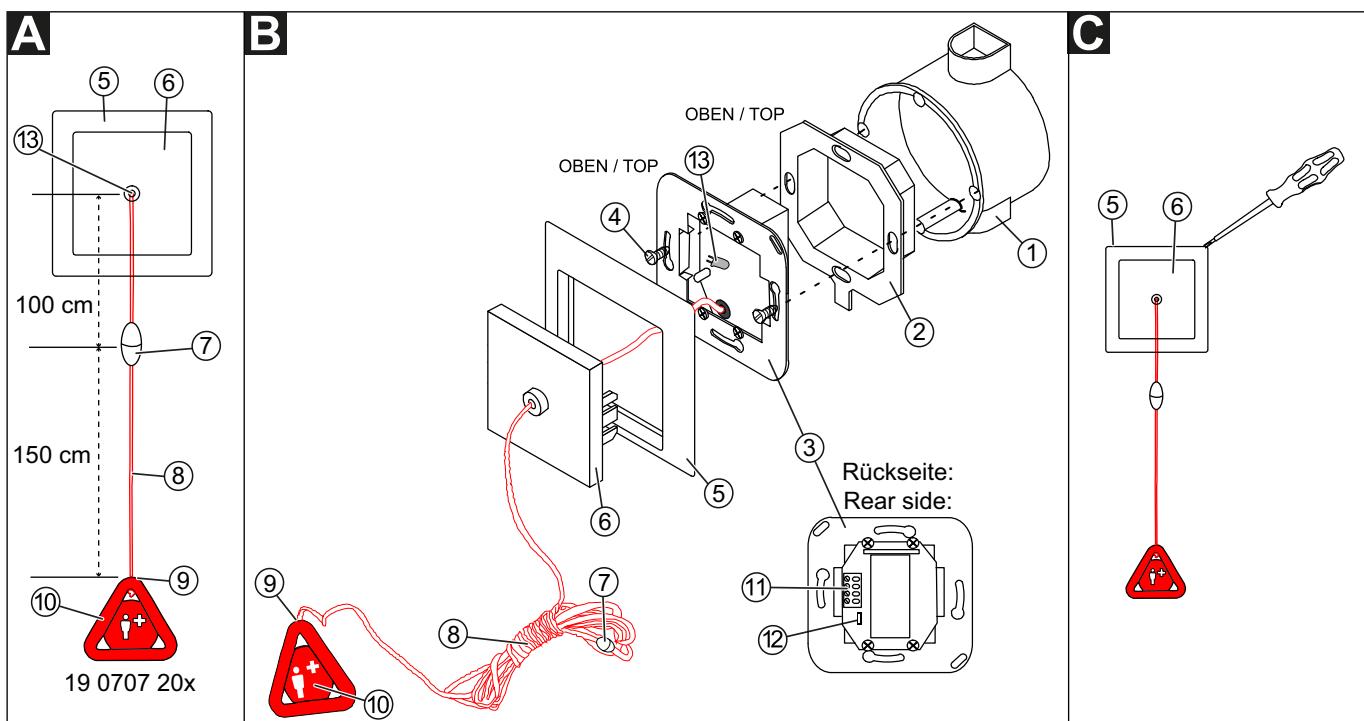
 **Note!** The complete installation of the system is described in the Technical Manual.

A Product description

- | | |
|-----------------------------|---|
| 1 - *Back box | 9 - Pull cord inlet |
| 2 - Sealing flange | 10 - Call handle |
| 3 - Pull cord switch insert | 11 - Connector |
| 4 - *Screws of the back box | 12 - Jumper |
| 5 - Frame | 13 - LED |
| 6 - Central plate | * Not included in the scope of delivery, please order separately. |
| 7 - Pull cord switch insert | |
| 8 - Pull cord | |

The pull cord [8] and the call handle [10] serve for triggering calls by pulling on the cord. The LED [13] lights up as soon as a call is triggered (reassurance light). The call type depends on which input of the room terminal the pull cord switch is connected to.

A safety release [7] in the pull cord [8] opens when the pulling force exceeds a limit value. The safety release is closed again by simply pushing it together.

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B Mounting

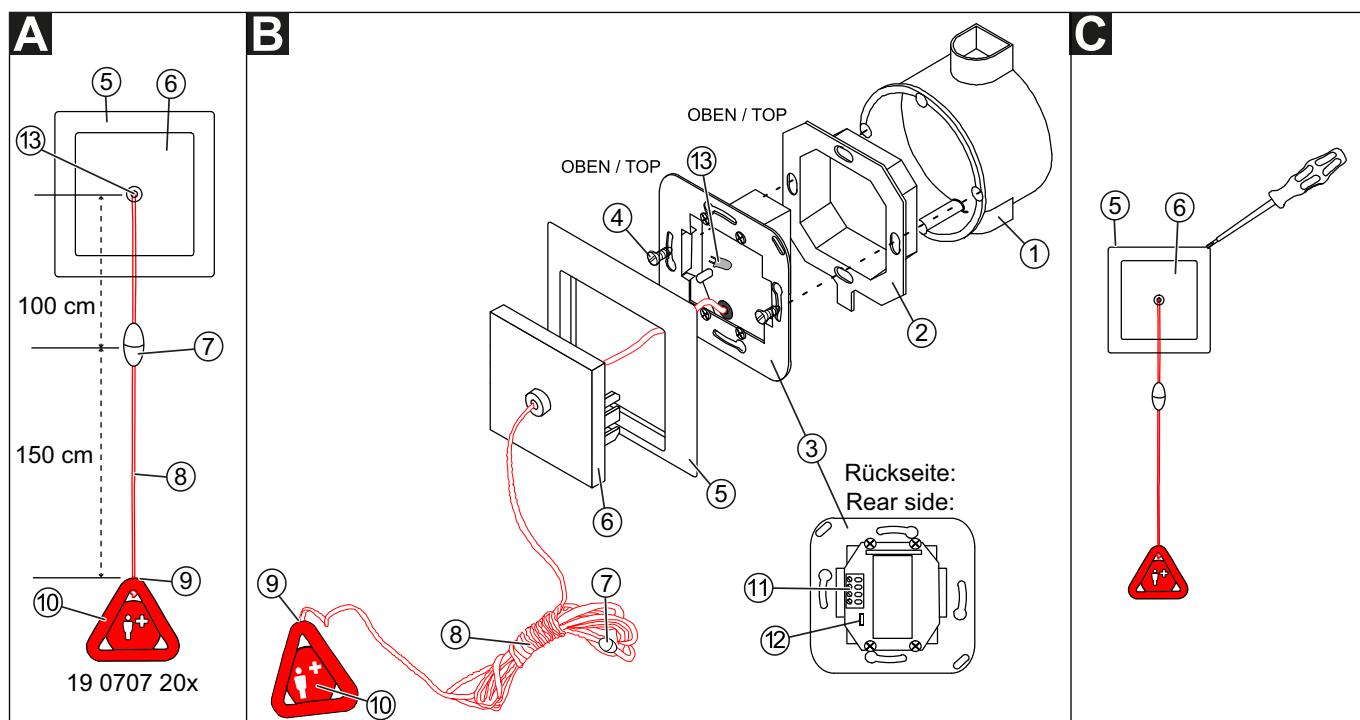
Caution! Pull cord switches in shower stalls must be installed at least 20 cm above the highest possible position of the shower head.

Ceiling or wall installation on one-gang back box:

- Strip the connecting cable to a suitable length. Remove the cable shield and drain wire up to the cable sheath. Strip the wires.
- Connect the wires according to the section **D** respectively **E** to the connector [11]. Note: The connector [11] can be pulled off temporarily.
- Screw the pull cord switch insert [3] and the sealing flange [2] with the screws [4] onto the back box [1]. When doing this, pay attention to the „TOP“ marking.
- Insert the central plate [6] into the frame [5].
- Press the central plate [6] together with the frame [5] onto the pull cord switch insert [3] until it engages audibly on both sides.
- The call handle [10] must be located between 10 and 20 cm above the floor. If necessary, shorten the pull cord [8] at the pull cord inlet [9] of the call handle [10] and re-knot it in the call handle [10].

C Dismantling

- Lever the frame [5] together with central plate [6] off the wall using a screw driver.
- Undo the screws [4] and remove the pull cord switch insert [3] with the sealing flange [2] from the back box [1].
- Unscrew the wires from the connector [11] on the rear side of the pull cord switch insert [3].

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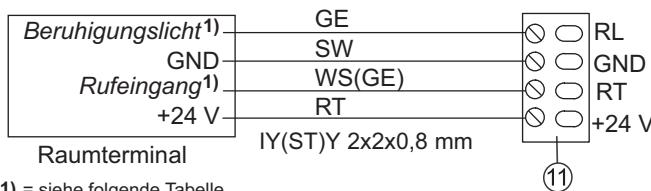
- Pull cord inlet
- Call handle
- Connector
- Jumper
- LED
- Not included in the scope of delivery, pls order separately.

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
EN - Installation Instructions

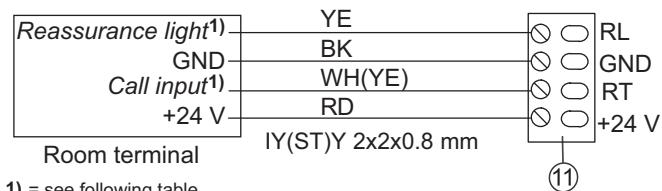■ Connection to a room terminal of order no. 19... (System CONCENTO^{PLUS})

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²⁾ Note! Many inputs of the room terminals are configurable, i.e. the call type can be changed. In addition there are further inputs available that are also configurable. For details, read the product leaflets accompanying the respective device.

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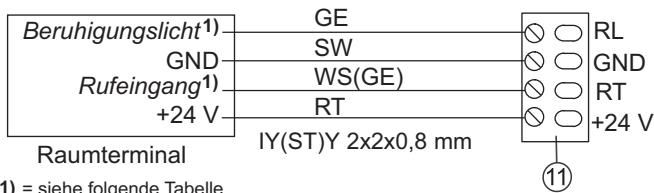
Caution! Do not remove the jumper [12]. It must always be inserted.

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
EN - Installation Instructions

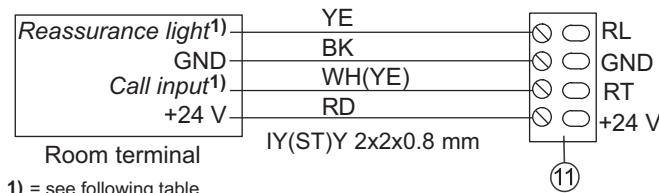E Connection to a room terminal of order no. 10... (System Concento)

| Raumterminal: | Rufart ²⁾ : | Beruhigungslicht ¹⁾ | Rufeingang ¹⁾ |
|--|------------------------|--------------------------------|--------------------------|
| Displaymodul (10 0700 80) RAB (10 0701 00) | WC-Ruf | WL | WR |
| | Ruf Bett 1 | RL | RT1 |
| | Ruf Bett 2 | RL | RT2 |
| Displaymodul (10 0700 85) RAB (10 0701 20) | WC-Ruf | WL | WR |
| | Ruf Bett 1 | RL | RT1 |
| | Ruf Bett 2 | RL | RT2 |
| | Ruf Bett 3 | RL | RT3 |
| Terminal (10 0700 x0) | WC-Ruf | WL | WR |
| | Ruf Bett 1 | RL | RT1 |
| | Ruf Bett 2 | RL | RT2 |
| | konfigurierbar | RL | R5 |
| | konfigurierbar | RL | R6 |

| Room terminal: | Call type ²⁾ : | Reassurance light ¹⁾ | Call input ¹⁾ |
|--|---------------------------|---------------------------------|--------------------------|
| Display module (10 0700 80) RAB (10 0701 00) | WC call | WL | WR |
| | Call bed 1 | RL | RT1 |
| | Call bed 2 | RL | RT2 |
| Display module (10 0700 85) RAB (10 0701 20) | WC call | WL | WR |
| | Call bed 1 | RL | RT1 |
| | Call bed 2 | RL | RT2 |
| | Call bed 3 | RL | RT3 |
| Terminal (10 0700 x0) | WC call | WL | WR |
| | Call bed 1 | RL | RT1 |
| | Call bed 2 | RL | RT2 |
| | configurable | RL | R5 |
| | configurable | RL | R6 |

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Terminating resistor

Several call devices on one input can be connected in line topology. Star topology is not permissible. The maximum cable length per input is 15 m.

The integrated 120 kOhm terminating resistor must be active in the case of the last call device at an input (= factory setting).

The integrated 120 kOhm terminating resistor must be inactive in the case of all other call devices at the input.

The terminating resistor is activated and deactivated via the jumper [12] (see Fig. B).

Jumper [12] set
= Terminating resistor active (factory setting)

Jumper [12] removed
= Terminating resistor not active

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