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Call switch with 2 connection sockets, order no. 19 0704 20

Red call button with reassurance light and integrated location light as LED. The reassurance light lights up as soon as a call is raised. Two equal sockets for connecting pear push switches or other plug-in call devices.

Suitable for the connection to the following room controls:

- Terminal, order no.: 19 0700 x0, 10 0700 x0
- Display module, order no.: 19 0700 80, 10 0700 8x
- RAB, order no. 19 0701 00, 10 0701 x0
- Control module ZL, order no. 19 0701 10

Note! The complete installation of the system is described in the Technical Manual.

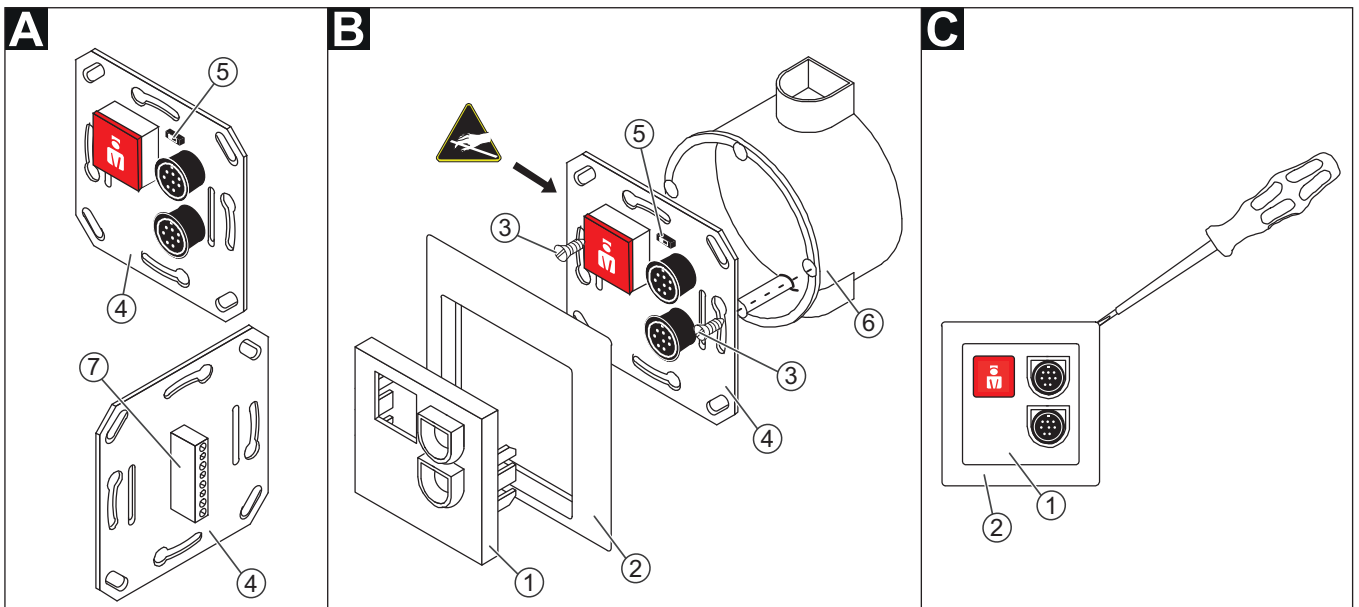
CAUTION! The printed circuit board includes electrostatic sensitive components. Avoid touching.

A Mounting

Wall mounting on one-gang back box:

1. Strip the connecting cable to a suitable length. Remove the cable shield and drain wire up to the cable sheath. Strip the wires.
2. Connect the wires to the connector [7] according to sections **D, E, and F**.
3. Screw the switch [4] to the back box [6] with the screws [3] of the back box.
4. Insert central plate [1] into the frame [2].
5. Press the central plate [1] together with frame [2] onto the switch until it engages audibly on both sides.

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- 1 - Central plate for 1 button & 2 connection sockets, order no. 19 9201 08 (19 9202 08 only for frame Busch-Jaeger future linear)
- 2 - *Frame, internal dimension: 55 x 55 mm
- 3 - *Screws of the back box

- 4 - Call switch with 2 connection sockets
 - 5 - DIP switch
 - 6 - *Back box, 1-gang
 - 7 - Connector
- * Not included in the scope of delivery, order separately.

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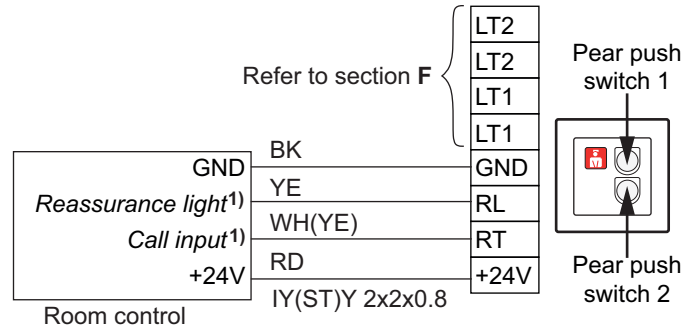
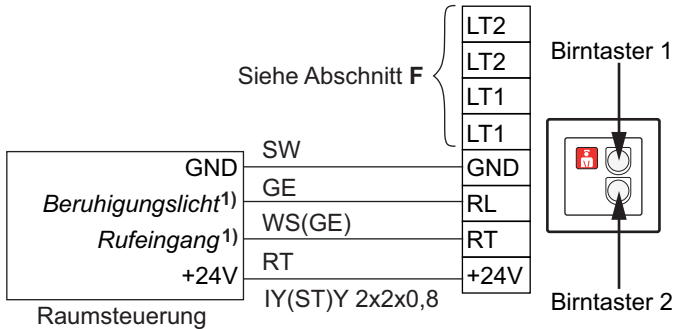
D Dismantling

1. Lever frame [2] and central plate [1] off the wall with a screw driver, see fig. C.
2. Undo the screws [3] of the back box and remove the switch [4].
3. Disconnect the wires from the screw connector on the rear of the switch [4].

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D Connection to a room control of order no. 19 07xx xx (System CONCENTO^{PLUS})

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²⁾ Note! Many inputs of the room controls are configurable, i.e. the call type can be changed. In addition there are further inputs available that are also configurable. For details, read the installation instructions for the respective room control.

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Several call devices in the room

Max. 4 call devices per input can be connected in line or star topology. The max. cable length per input is 20 m. The integrated 120 kOhm terminating resistor must be active for all call devices (= factory setting).

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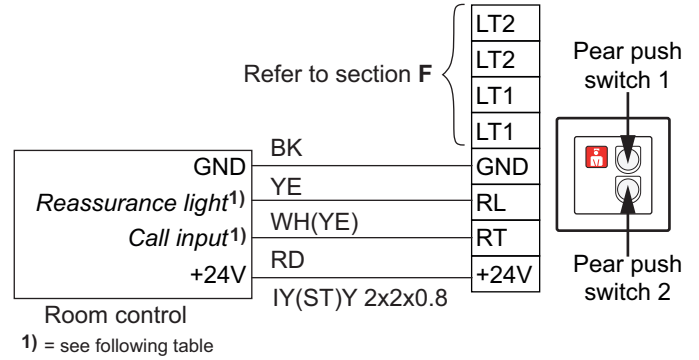
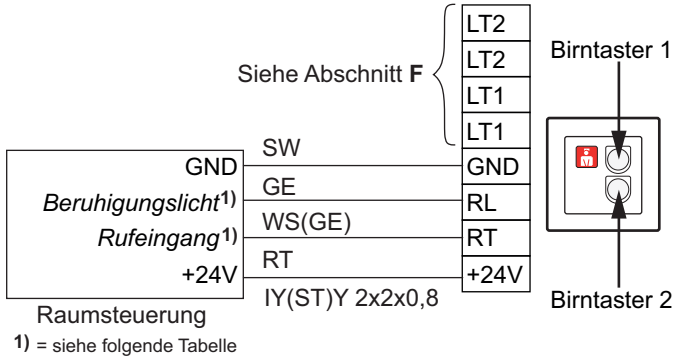
Caution! Do not change DIP switch [5]. It must always be set to ON.

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☒ Connection to a room control of order no. 10 07xx xx (System Concerto)

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Several call devices on one input can be connected in line topology. Star topology is not permissible. The maximum cable length per input is 15 m.

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The integrated 120 kOhm terminating resistor must be active in the case of the last call device at an input (= factory setting).

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The integrated 120 kOhm terminating resistor must be inactive in the case of all other call devices at the input.

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The terminating resistor is activated and deactivated via the DIP switch [5] (see Fig. A).

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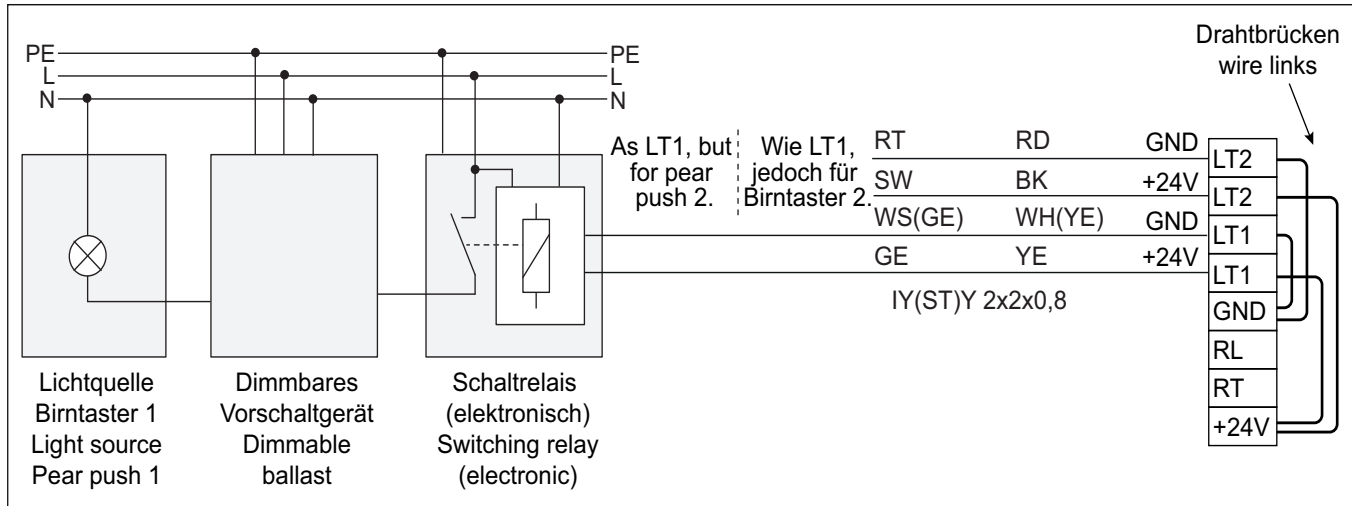
Terminating resistor active = DIP switch ON (left)

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Terminating resistor not active = DIP switch OFF (right)

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Outputs LT1 and LT2 provide an output voltage of 24 V DC and a current of max. 200 mA as long as the light button on the pear push switch is pressed.

To select a relay and connect other relays, read the chapter entitled "Light Control" in the Technical Manual.

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Technical data

Nominal voltage	24 V DC
Standby current consumption	9.1 mA (without using the connection sockets)
Max. current consumption	18.5 mA (without using the connection sockets)
Rated wiring	0,2 – 2.5 mm ²
Insulation strip length	7 ± 0.5 mm
Dimensions (HxWxD)	71 x 71 x 20 mm
Weight incl. central plate and frame	approx. 55 g
Ambient temperature	+5 °C – +40 °C
Relative humidity	0 % – 85 %