Pneumatiktaster-Einsatz

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Pneumatic switch insert

Intended for connection to the room bus of a CONCENTO^{CARE} nurse call system.

NOTE! The complete installation of the system is described in the Technical Manual.

A Product description

A call or a WC call is triggered by pressing the red rubber ball [9], which is connected to the pneumatic switch insert [6] via an air hose [8].

Also an additional red call button [7] serves for triggering calls. An LED in the call button lights up faintly to locate the button in the dark (location light). The LED lights up brightly as soon as a call is triggered (reassurance light).

- [1] *Back box, 1-gang
(solid wall: 17 0100 00,
partition wall: 17 5100 00)
- [2] Mounting plate
- [3] *Back box screws
- [4] *Cover frame, 1-gang

- [5] Terminal block
(plug-in terminals)
- [6] Pneumatic switch insert
- [7] Call button (red)
- [8] Air hose (2 m) made of silicone
- [9] Rubber ball

* Not included in the scope of delivery, please order separately.

B Mounting

Wall installation on one-gang back box:

- Bolt the mounting plate [2] to the back box [1]. The four square recesses in the mounting plate are located at the top and bottom.
- Prepare the wires and feed them through the mounting plate [2] (see section "Connection").
- Connect the pneumatic switch insert [6] according to section "Connection".
- Hold the cover frame [4] in front of the mounting plate on the wall.
- Plug the pneumatic switch insert [6] onto the mounting plate [2] so that the latching hook of the pneumatic switch insert fits into the square recesses of the mounting plate [2]. The call button is located on top.
- Ensure that the air hose [8] is firmly plugged onto the nozzle on the pneumatic switch insert [6].

C Dismantling

- Pull the cover frame [4] together with the pneumatic switch insert [6] off the mounting plate [2].
- For disconnecting the wires, refer to fig. E.

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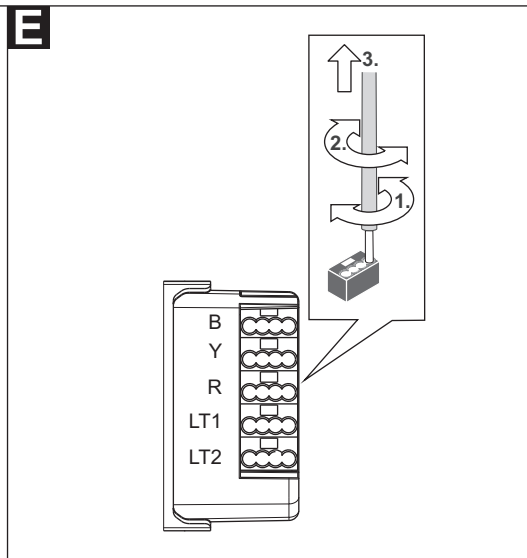
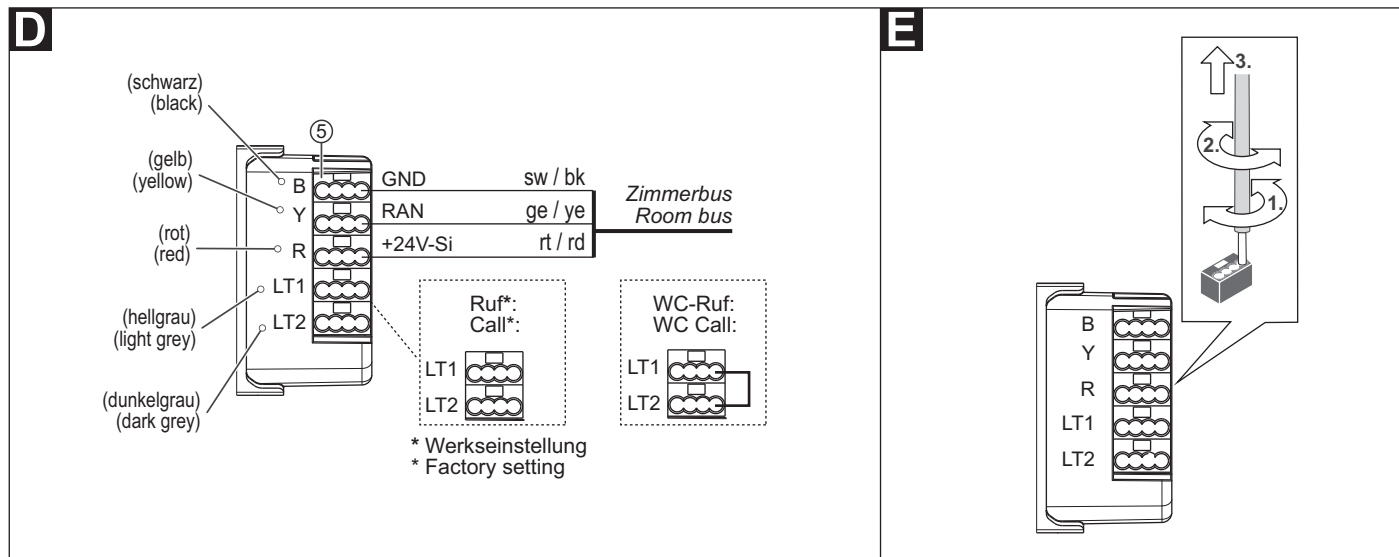
EN - Installation Instructions

D Connection

1. Pull the connection cable out through the mounting plate and strip it to a suitable length. Remove the cable shield and drain wire up to the cable sheath. Strip the wires.
2. Connect the wires according to the connection diagram.

Note: The terminal block [5] can be pulled off temporarily.

3. If the pneumatic switch insert is to be used for triggering WC calls, a wire bridge must be fitted between connection points LT1 and LT2. No wire bridge is to be set for triggering standard calls (= factory setting).

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Technical data

Nominal voltage	24 V=
Standby current consumption	14 mA
Room bus connection	Cable type: J-Y(St)Y 2x2x0.8
Skinning length	6 mm
Installation depth	24 mm
Degree of protection	IP 22
Ambient temperature	+5°C – +40°C
Relative humidity	0 % – 95 %