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Materialien am Gerät

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Cleaning/Disinfection

Materials on the device

When selecting the cleaning agent and disinfectant, consider the properties of the device materials:

- ① Call button (red): ASA/PC and TPE-S “Solplast”
- ② Touch field (black): PET film
- ③ Connection cable: Soft PVC
- ④ Housing shell (white): ASA/PC
- ⑤ Bracket: ASA/PC

General Information

Basically, it is safe to wipe the unit and the bracket with a damp cloth suitable for sensitive surfaces using a diluted, lukewarm solution of detergent and water.

Make sure that no moisture enters the headphone jack.

No detergent residues must remain on the unit, e.g. in the speaker gap.

Observe the applicable hygiene regulations for your hospital.


Disinfectants

When testing disinfectant compatibility, products containing ethanol, such as Bacillol® 30 Sensitive Tissues (Bode Chemie) and Suma Alcohol Wipes (Diversey), were found to be inconspicuous.

Wiping the touch field during operation

Before you clean the touch field, you must temporarily disable it. Press and hold the CLEAN icon [6] for 3 seconds until all LEDs illuminate. When two LEDs are lit (approx. 9 sec.) the touch field has no function and it is safe to clean it.

Tunstall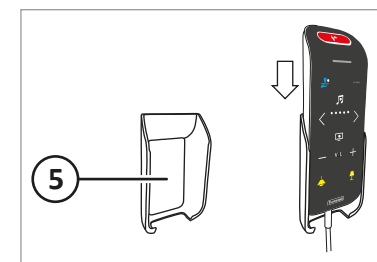

00 8804 40, 09/23 (Rev 5.0)

ePat®lite
77 0370 00, 77 0370 10

Reinigung/Desinfektion
Cleaning/Disinfection