

How calls are displayed

	Alarms: Blue is flashing very quickly. Very fast sequence of sounds.
	Emergency calls: Red is flashing quickly. Fast sequence of sounds.
	Calls: Red is flashing slowly. Slow sequence of sounds.

	Fresh calls flash. The whole field displayed in colour.
	Answered calls do not flash. Only the title bar displayed in colour.


How to answer a call




- Tap the call.

✓ A speech connection to the call location is established.

- Tap the arrow to open the call details view.

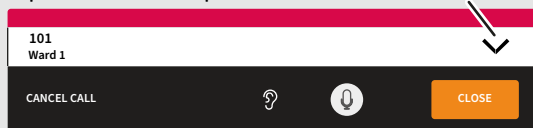

✓ The call details are displayed:

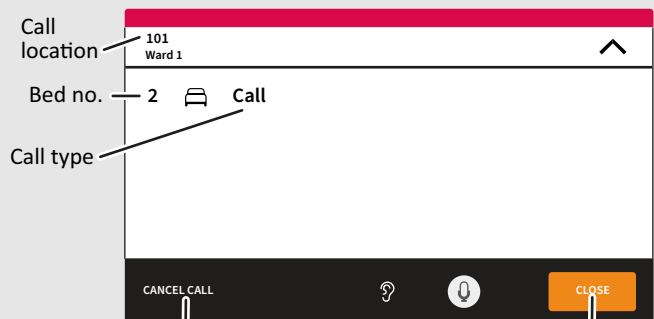


Cancel the call
(only for call type "Call")

Close the
speech connection

- Speak to the caller and ask what help is needed.
- Tap CLOSE to close the speech connection.
- Provide the required help.


Home screen

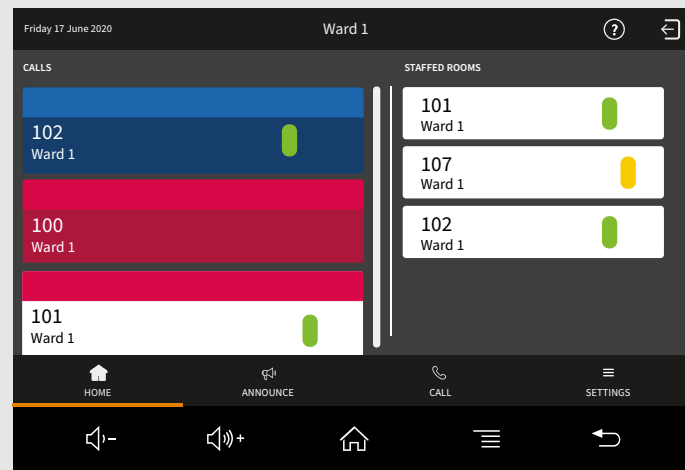

Fault in the nurse call system! Immediately inform a technician, if this icon appears in the upper bar!


Log out! Log out, when you leave the nurse station.

How to speak to staff

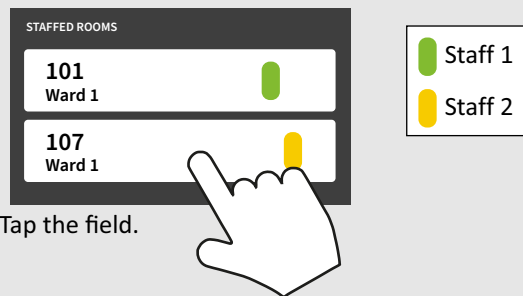

- Tap the field.

✓ A speech connection to the staff is established:

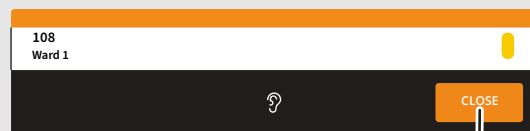

Close the speech connection


Status of a speech connection

- Privacy in the room**
You cannot listen to what is said in the room.
- No privacy in the room**
You can listen to what is said in the room.
- Your microphone is turned on.**
The person in the room can hear you.
- Your microphone is turned off.**
The person in the room cannot hear you.
- Volume up**
by tapping the icon.
- Volume down**
by tapping the icon.




NOTE! For a detailed user guide, tap the question mark on the Home screen.

How to make an announcement

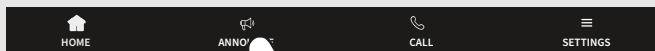

- Tap ANNOUNCE.

✓ Available announcement targets are displayed:

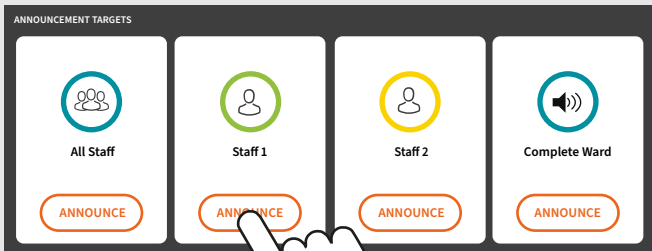


- Tap ANNOUNCE.

✓ The speech connection for the announcement is established.


- Make your announcement.
- Tap END to finish the announcement.

How to call

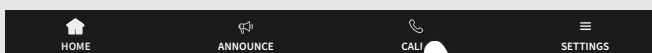

- Tap CALL.

✓ Possible call destinations are listed.

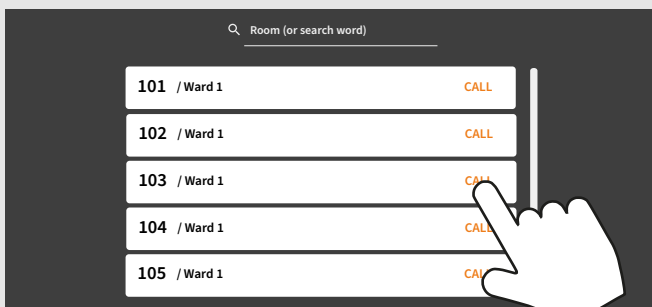



- Tap CALL.

✓ The speech connection is established:




- Talk to the person on the other end.
NOTE! If the privacy function at the other end is on, ask the patient to press the call button to turn the privacy off.
- Tap CLOSE to close the speech connection.


Ward coupling

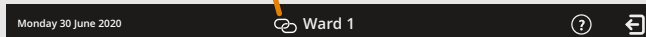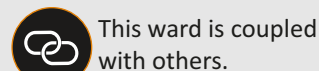

How to activate or deactivate a ward coupling

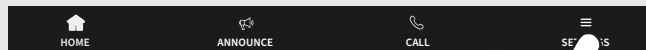

- Tap SETTINGS.
- Tap "Ward coupling".


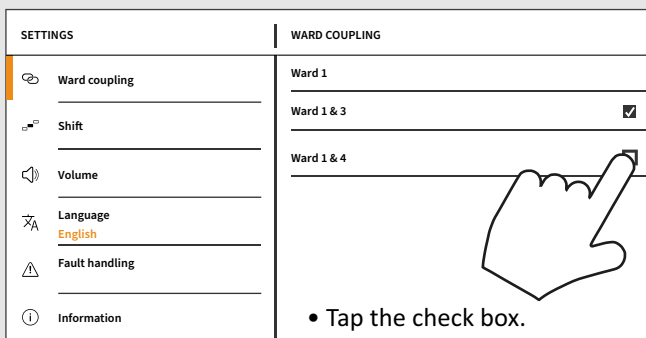


- Tap the check box.


NOTE! More than one ward coupling can be active simultaneously.

- Tap HOME to end the process.


Shift operation

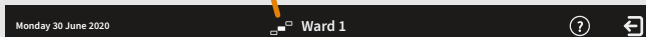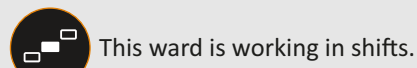

How to activate and deactivate a shift

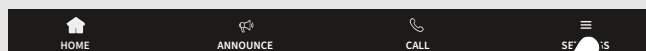

- Tap SETTINGS.
- Tap "Shift".

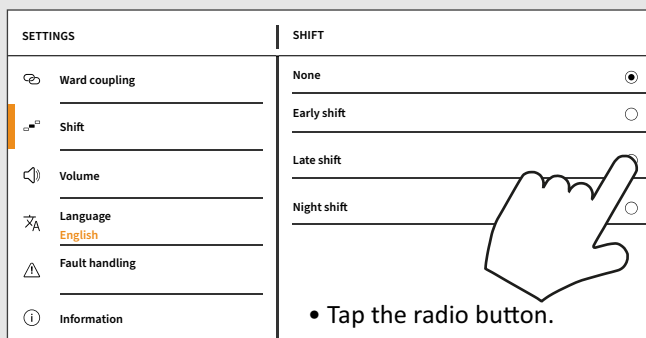

- Tap the radio button.

- Tap HOME to end the process.