Telephone transmitter BE1431

DE	Benutzerhandbuch	3
DK	Brugervejledning	9
EN	User manual	15
ES	Manual del usuario	21
FI	Käyttöopas	27
FR	Mode d'emploi	33
NL	Gebruiksaanwijzing	39
SE	Bruksanvisning	45
HU	Használati utasítás	51
EN	Installer's guide	58

Installation

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

Trigger input

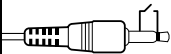You can use both stereo and mono plugs. The 3.5 mm input is located on the top right side and the 2.5 mm input under the lid.

Mono

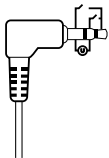Trig is made by closing the two connectors

Stereo

Trig is made by:

- 1 Closing the outer and middle connector or closing the outer and inner connector
- 2 Applying 2-30VDC or 3-24VAC 5-150Hz between the middle and inner connector

Please note

When you use the 2.5 mm ext trig, the event must last for more than 2 seconds

Technical information

Power supply	Battery power: 2 x 1.5 V AA (LR6) lithium or alkaline
Power consumption	Active < 70 mA, Idle position < 15 μ A
Operating time	Alkaline ~ 5 years, Lithium ~ 10 years
Activation	Via test buttons, RJ11 input to analogue telephone network (26 - 120 V RMS, 15 - 100 Hz), 2.5 mm ext trig, 3.5 mm ext trig
Radio frequency	315 MHz, 433.92 MHz or 868.3 MHz depending on region
Coverage	50 – 250 m depending on radio frequency. Coverage is reduced by walls, large objects and other radio transmitters.
Number of radio keys	64 as standard. Up to 256 with optional software.
Accessories	BE9250 Mobile phone sensor, BE9024 Contact mat BE9023 Magnetic switch and BE9253 External trig cable

Additional information

Usage and storage	For indoor use only. Temperature: 15°-35° C. Relative humidity: 5%-95%.
Maintenance and cleaning	Maintenance free. Clean with a dry cloth. Don't use household cleaners, aerosol sprays, solvents, alcohol, ammonia, or abrasives. Not reusable.
Warranty and service	If your telephone transmitter appears to be damaged or doesn't function properly, follow the advice in this booklet and the online resources. If it still doesn't function properly, contact your retailer for information about obtaining warranty service.

Hereby, Bellman & Symfon declares that this Telephone transmitter is in compliance with the essential requirements and other relevant provisions of directives:

R&TTE 1999/5/EC, LVD 1973/23/EC,
EMC 1989/336/EC, MDD 1993/42/EEC.

Use and disposal of batteries

Replace only with the same or equivalent type recommended by the manufacturer. Please dispose of old, defective batteries in an environmentally friendly manner in accordance with the relevant legislation in your country.

FCC compliance statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Warning: Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interfer-

ence in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is con-

nected.

- Consult the dealer or an experienced radio/TV technician for help.

For a complete Declaration of Conformity please contact the Bellman & Symfon European office.

Industry Canada statement

This device complies with Industry Canada Licence-exempt RSS-210. Operation is subject to the following two conditions:

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(1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device.

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This Class B digital apparatus complies with the Canadian ICES-003.

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Bellman & Symfon Europe AB
Bellman & Symfon Asia Ltd.